coltène 
whaledent®

INSTRUCTIONS
HYGENIC
COLD CURE DENTURE ACRYLIC

FOR PROFESSIONAL DENTAL USE ONLY

TYPE II

CLASS I

HYGENIC COLD CURE DENTURE ACRYLIC possesses the same smooth working properties as **HYGENIC ACRYLIC** (heat curing) and is mixed and packed much the same.

MIXING – 3 parts powder and 1 part liquid by volume. Mix as usual, spatulate as required, and knead to a smooth dough.

PACKING – Pack when material is no longer sticky but before it begins to feel dry and grainy.

WORKING TIME – After mixture has reached proper packing consistency, the dough will remain sufficiently soft to pack for about 5 minutes at room temperature of 75° F. This working time will be somewhat longer at lower temperature and shorter at higher temperature. Once the dough is ready to pack, it should be packed, trial packed and the final closure made as quickly as possible.

CURING – After final closure, leave case in flask under pressure for at least 1 hour. (While polymerization is practically complete in 20 minutes, the longer time in the flask insures a more perfect cure.)

HYGENIC COLD CURE DENTURE ACRYLIC may also be used for repairs.

REPAIRS – Make the usual preparation. If a cast is to be used, paint with separator and allow to dry. When ready, coat surface to be repaired with Hygenic Cold Cure Liquid and pack the cold cure dough (mixed 3 parts powder to 1 part liquid) into the preparation while sticky.

Place in pressure curing pot with warm water for 20 minutes at 20 psi.

For repairs by the powder-liquid method, use
HYGENIC SELF-CURING REPAIR ACRYLIC.

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CAUTION: Federal law restricts this device to sale by or on the order of a dentist.

CAUTION: FLAMMABLE. Contains Methyl Methacrylate monomer (Flash Point 50°F). Keep away from heat, spark and open flame.

AVOID INHALATION OR INGESTION: Use in well ventilated area. Keep container closed when not in use.

AVOID PROLONGED OR REPEATED CONTACT WITH SKIN: Wash skin with soap and water after contact.

KEEP OUT OF REACH OF CHILDREN

Coltène /Whaledent Inc.

235 Ascot Parkway
Cuyahoga Falls, OH 44223 / USA
Tel. USA & Canada 1 800 221 3046
+1 330 916 8800

Fax +1 330 916 7077
info@coltenewhaledent.com

Authorized EU Representative:

Coltène /Whaledent GmbH + Co. KG

Raiffeisenstrasse 30
89129 Langenau / Germany
Tel. +49 (0) 7345 805 0

Fax +49 (0) 7345 805 201
productinfo@coltenewhaledent.de

Made in USA

P/N-86787B