



| Resin | | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|---------------------|---------------------------------|------------------|------------------|-------------------|-----------|
| Cast Crown | Pro Cast Denture | 5 | 13 | 8 | ON |
| Guide | | 6 | 10 | 8 | ON |
| Tray | | 7 | 13 | 8 | ON |
| Splint | | 10 | 6 | 8 | ON |
| Model Temp | Model Signature Pro Denture IBT | 10 | 13 | 8 | ON |
| Pro Splint | | 15 | 8 | 1 | ON |
| Element Model Beige | Element Model White | 20 | 2 | 5 | OFF |
| Gingiva | | 20 | 13 | 8 | ON |



| Resin | | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|-------|-------|------------------|------------------|-------------------|-----------|
| Guide | | 5 | 16 | 10 | ON |
| Tray | | 7 | 16 | 10 | ON |
| FLEX | Model | 10 | 16 | 10 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|---------------|------------------|------------------|-------------------|-----------|
| Model Temp | 10 | 13 | 8 | ON |



| Resin | | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|-------|--------------------|------------------|------------------|-------------------|-----------|
| Teeth | Temp Crown plus | 2min 30s* | 13 | 8 | ON |

*** Flip the object and restart the post curing cycle**

| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|---|------------------|------------------|-------------------|-----------|
| Crown & Bridge Denture base Denture teeth Try-in | 10 | 13 | 8 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|--|------------------|------------------|-------------------|-----------|
| Cast Lumina Zero Denture Cast NF | 5 | 13 | 8 | ON |
| Guide | 6 | 10 | 8 | ON |
| Tray | 7 | 13 | 8 | ON |
| Splint | 10 | 6 | 8 | ON |
| IBT Temp Match Model Grey Mock up Model GB Model Ivory Ortho Model Laviva light-pink | 10 | 13 | 8 | ON |
| Clara | 15 | 8 | 1 | ON |
| Gingiva | 20 | 13 | 8 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|---|------------------|------------------|-------------------|-----------|
| Tray | 7 | 13 | 8 | ON |
| Gingiva Model Beige Model Grey Model Transparent | 10 | 13 | 8 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|---------------------------------------|------------------|------------------|-------------------|-----------|
| Guide Model2 beige Model2 white | 10 | 13 | 8 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|--|------------------|------------------|-------------------|-----------|
| Ocean Blue | 5 | 13 | 8 | ON |
| Beluga White Orca Black Dolphin Grey | 10 | 13 | 8 | ON |

| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|---|------------------|------------------|-------------------|-----------|
| KeyGuide Key Try In | 5 | 16 | 10 | ON |
| KeyMask KeyModel KeyModelUltra Ivory KeyModelUltra Grey KeyOrtho IBT KeyOrthoModel KeySplintSoft KeyTray | 10 | 16 | 10 | ON |
| KeySplint Hard | 12 | 8 | 5 | ON |
| KeySplint Hard Clear | 10 | 8 | 5 | ON |
| KeySplint Soft Clear | 10 | 10 | 10 | ON |

| Resin | | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|-------------|---------|------------------|------------------|-------------------|-----------|
| Cast | Denture | 5 | 13 | 8 | ON |
| Guide | | 6 | 10 | 8 | ON |
| Tray | | 7 | 13 | 8 | ON |
| Splint | | 10 | 6 | 8 | ON |
| Model GB | Temp | 10 | 13 | 8 | ON |
| Ortho Model | IBT | | | | |
| Gingiva | | 20 | 13 | 8 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|---------------------------|------------------|------------------|-------------------|-----------|
| CMFH Denture 3D+ | 5 | 13 | 8 | ON |
| Tray | 7 | 13 | 8 | ON |
| SG Ortho IBT Model 2.0 | 10 | 13 | 8 | ON |
| Ortho Flex | 10 | 16 | 10 | ON |

NYTE 3D

| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|--------------------------|------------------|------------------|-------------------|-----------|
| Cast Press Model Sand | 10 | 13 | 8 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|--|------------------|------------------|-------------------|-----------|
| Splint | 10 | 16 | 10 | ON |
| Sculpture SG | 20 | 9 | 2 | ON |
| Bite Tray Denture Try in IBT Model | 20 | 12 | 10 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|----------|------------------|------------------|-------------------|-----------|
| CROWNTEC | 3* | 16 | 10 | ON |

*** Flip the object and restart the post curing cycle**



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|--|------------------|------------------|-------------------|-----------|
| Cast & Press | 5 | 13 | 8 | ON |
| Tray | 7 | 13 | 8 | ON |
| Gingiva Model Plus UV Desert Model UV Smoky white Model Plus UV Stone | 10 | 13 | 8 | ON |



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|--|------------------|------------------|-------------------|-----------|
| c&b temp | 1min30s* | 16 | 10 | ON |
| Cast | 9 | 13 | 8 | ON |
| Dentbase Model 2.0 SG Splint comfort | 7 | 13 | 8 | ON |
| Model Beige | 5 | 16 | 10 | ON |
| Model Fast Splint Tray Try-In | 5 | 8 | 10 | ON |

*** Flip the object and restart the post curing cycle**



| Resin | (T) Time(min) | (P) Intensity | (D) Duty Cycle | (B) BL |
|--|------------------|------------------|-------------------|-----------|
| VeriModel Golden Brown VeriModel Grey VeriModel Ivory VeriModel White | 5 | 16 | 10 | ON |
| VeriTray | 7 | 13 | 8 | ON |
| VeriCast VeriSplint VeriGuide | 10 | 16 | 10 | ON |