Manufacturer  
SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

EC REP

SHOFU DENTAL GmbH  
Am Brüll 17, 40878 Ratingen, Germany

01:2019-03  
300030317

EN

READ CAREFULLY  
BEFORE USE

AcryPoint-E are used for adjusting, finishing and polishing acrylic dentures or other acrylic appliances as well as plastic impression trays. Use AcryPoint-E "Coarse" (dark grey) for adjustment, "Medium" (brown) for finishing and "Fine" (light grey) for polishing.

#### PRECAUTIONS

- Never exceed the maximum permissible rotation speeds.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

#### STERILIZATION

- Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization.
- When necessary, remove any contaminants with cleaning liquid or disinfectant, then sterilize.
- Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion.
- Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

#### MAXIMUM PERMISSIBLE ROTATION SPEED

AcryPoint-E	15,000 min <sup>-1</sup>
-------------	--------------------------

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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