DURA-POLISH & DURA-POLISH DIA

ENGLISH

Instructions for Use

DURA-POLISH is Al_2O_3 particle impregnated polishing wax for prepolishing and polishing. **DURA-POLISH DIA** is a diamond particle impregnated polishing wax for super polishing.

Make **DURA-POLISH** or **DURA-POLISH DIA** adhere to brush or felt wheel with slow revolution speed and, by lightly pressing brush or felt wheel against the restorations, polishing can be easily done.

Type	DURA-POLISH	DURA-POLISH DIA
Application	Hard resin, Metal, Denture resin	Hard resin, Porcelain
Effect	Prepolishing and finish-polishing	Super polishing for luster

Caution

- Use this product with less than 10,000 min-1 revolution speed since high-speed revolution may cause spattering of the polishing wax.
- Do not strongly press when adhering this product to brush, etc. in order to avoid splash out from the container.
- Use local exhaust fan and dust-proof.
- Wear eye-protecting glasses when operating.
- Use this product for indicated purpose only.

Packaging

DURA-POLISH: jar 20 g
DURA-POLISH DIA: jar 5 g

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Lieferform

DURA-POLISH: Dose 20 g
DURA-POLISH DIA: Dose 5 g

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Envasado

DURA-POLISH: frasco 20 gr.
DURA-POLISH DIA: frasco 5 gr.

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DURA-POLISH: frasco de 20 g

DURA-POLISH DIA: frasco de 5 g

SHOFU INC., 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605 - 0983, Japan.
SHOFU INC., Singapore Branch Bldg. 215 Henderson Road # 03-01, Singapore 159554.
SHOFU INC., Shanghai Rep. Office, 1006 Room, Hi-Tech Building, 900# Yishan Road, Caohejing Development Park, Shanghai 200233, China
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, CA 92069, USA.
SHOFU DENTAL PRODUCTS LTD. Duke's Factory, Chiddingstone Causeway, Tonbridge, Kent, TN11 8 JU, UK.
SHOFU DENTAL GmbH, Am Brühl 17, 40878 Ratingen, Germany, EU REPRESENTATIVE.

06/2004