

instructions. This label should be applied to the bottom of the lower member so that matched upper and lower members are used together.



**LABEL HERE**

## **Denar® FlexTrack**

Provides the flexibility of movement in both open and closed condylar tracking on one articulator. For use on Denar Mark 320 and Mark 310 articulators.

Call us to find out how quick and easy this product is to use.



RESTORATIVE ORAL HEALTH  
Whip Mix Corporation  
361 Farmington Avenue  
Louisville, KY USA 40209  
Toll-Free: 800-626-5651  
Phone: 502-637-1451  
Fax: 502-634-4512  
[www.whipmix.com](http://www.whipmix.com)

© 2012 Whip Mix Corporation  
FN 20001046-F AA 1211



## **Denar® Mark 300 Series Dust Cover**



**Keeps dust out of the  
centric latch mechanism**

## Installation:

---

The Denar® Mark 300 Series Dust Cover can be installed in just a few minutes. Remove the upper member by releasing the centric latch and elastic retainers from the shoulder screws.

The dust cover clips onto the upper member and covers the pivot bar which contains the centric latch mechanism. Note that there is a slot in the dust



cover which aligns with the centric latch lever and allows it to travel freely. Grip the upper member firmly with one hand. Holding the dust cover with the other hand, align the slot with the latch lever and firmly push the cover onto the pivot bar.



## Important note:

---

All Mark 300 Series articulators are factory matched, calibrated and labeled with serial numbers. It is essential that only matched upper and lower members be used together. Because the Mark 300 Series Dust Cover may obscure the serial number on the upper member, a new serial number is etched onto the dust cover. A label with the same serial number is included with these

