


Cadiax® Muscle Deprogrammer Instructions for Use.

This device may be used to deprogram tight muscles before beginning a tracing. It will also provide a patient with a place to rest their maxillary teeth while recording excursive movements.

1. Attach the deprogrammer by snapping it to the tray clutch stem, resting it on the top surface of the tray clutch.
2. For additional security, you may choose to place a small amount of poly-vinyl siloxane material on the underneath side of the deprogrammer where it will rest on the surface of the clutch.

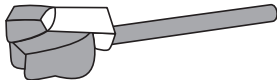

RESTORATIVE ORAL HEALTH

Whip Mix Corporation - West
1730 East Prospect Rd., Suite 101
Fort Collins, CO 80525

Toll-Free: 800-201-7286 • Fax: 1970-472-1793

www.whipmix.com