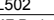
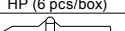
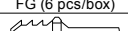
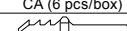
SYSTEM

	Contouring		Finishing				Polishing		Super Polishing		Luster Polishing		
Disks	Black - Coarse (50 pcs/box)		Violet - Medium (50 pcs/box)				Green - Fine (50 pcs/box)		Red - Super Fine (50 pcs/box)		SuperBuff (50 pcs/box)	Buff Disk (25 pcs/box)	
	 L506	 L507	 L508	 L528	 L509	 L522	 L501	 L519	 L502	 L521	 0536	 L523	 L524
							X-TREME Green - Fine (50 pcs/box)		X-TREME Red -Super Fine (50 pcs/box)				
							 L511	 L531	 L512	 L532			
Polystrips	Black/Violet - Coarse/Medium (100 pcs/box)						Green/Red - Fine/Super Fine (100 pcs/box)						
	 L525						 L526						
Mandrels	Metal mandrel						Plastic mandrel						
	HP (6 pcs/box)			FG (6 pcs/box)			CA (6 pcs/box)			CA (100 pcs/box)			
	 0437			 0438			 0439			 0440			

EN READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

Super-Snap is an easy to use, color co-ordinated, 4-step disposable disk system with metal-free center, designed for contouring, finishing and polishing of all types of direct aesthetic resins (microfilled, micro-hybrid, nano and nano-hybrid). Super-Snap Polystrips are doubled-ended interproximal strips with the same color code as the Super-Snap Disks and incorporate two abrasive grits per strip for finishing and polishing of the interproximal restoration surface.

INDICATIONS FOR USE

Finishing and polishing of all types of direct aesthetic resin materials

PRECAUTIONS AND WARNINGS

- Do not use damaged, defective or worn instruments/mandrels, i.e. corrosion, surface degradation due to repeated use.
- Use protective mask and eyewear when operating handpieces.
- Avoid contacting soft tissues to avoid injury during use.
- Super-Snap Disks and Polystrips are designed for single use only. Discard the Super-Snap Disks and polystrips after use in accordance with the disposal protocol of your healthcare facility.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- The product is intended for use by dental professionals only.

DIRECTIONS FOR USE

- Mount the disk securely on the dedicated Super-Snap mandrel.
- Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
- Apply uniform, steady pressure for finishing and feather light pressure for polishing. Unidirectional movement of the disks prevents formation of secondary scratches and a visible step at the restoration-tooth junction.
 - Follow the recommended finishing and polishing sequence with the color coded disks for the best results. Reducing or skipping the recommended steps may compromise the quality of the final polish.
 - Use saliva ejector or suction to eliminate moisture and debris during the procedure. Use protective mask and eyewear while using the product.
 - Avoid exceeding maximum permissible speed for each abrasive.
- Product name | Maximum permissible rotation speed | Water spray
Super-Snap Disks | 15,000 min⁻¹ | No
- Clean the debris from restoration surface after each finishing/polishing step.
- After using dedicated metal mandrel, remove any contaminants immediately, disinfect and sterilize. Dispose the Super-Snap Disks after each use.

Super-Snap Polystrips

Insert the abrasive free center of Super-Snap Polystrip in the interproximal area. Follow the same color coded abrasive grit as Super-Snap Disks. Adapt over the restoration surface to be finished and polished and apply uniform pressure in back and forth motion, starting with the coarse section followed by the finer section of the strip. Eg. Black followed by Violet and then, Green followed by Red.

- Do not reuse Super-Snap Polystrips.
- Exercise caution during use near the cervical area to avoid injuring the gum with the edge of the Polystrip.
- Avoid bending the strip at sharp angles.

STORAGE AND STERILIZATION

Super-Snap Disks and Polystrips should be stored away from direct sunlight, moisture and corrosive chemicals/vapor. Sterilize Super-Snap metal mandrels before use. Remove any contaminants using suitable cleaning liquid or disinfectant and sterilize.

Product name	Autoclave
Super-Snap metal mandrel	134 °C (273 °F) – 3'00"
	or
	121 °C (250 °F) – 30'00"

Follow manufacturer's instructions for use of cleaning liquid, disinfectant or sterilizer. Chloride disinfectant should be avoided.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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