

InstRenew® Sharpening Assistant

Implementing an instrument maintenance program in your practice is the best way to protect your instrument investment. With proper care and periodic maintenance your Nordent instruments will provide you with years of trouble-free service and will enable you to perform the best possible patient care.

Keeping your hygiene scalers and curettes sharp is made simple and efficient by using the Nordent InstRenew® Sharpening System. Nordent also offers a complete selection of sharpening supplies and a professional sharpening service to help you get the most from your instrument investment.

SHARPENING INSTRUMENTS

INSTRENEW SHARPENING ASSISTANT	2
ACCESSORIES	3
SHARPENING STONES	4



INSTRENEW® SHARPENING ASSISTANT


InstRenew is the only instrument sharpening device developed by instrument makers to make it EASY FOR ANYONE to sharpen ANY BRAND and ANY PATTERN of Scalers and Curettes with professional results every time. The patented design makes it possible to achieve perfect factory blade angles without the guesswork and frustration that are common with other sharpening methods. If you have been looking for a cost effective, fast and easy way to improve your in-house sharpening program that will get everyone in your practice sharpening with the exact same angle every time...then InstRenew is the answer! See a complete video demonstration at www.nordent.com.

InstRenew Sharpening Assistant comes complete with everything you'll need to begin sharpening with professional results today. Includes:

- Diamond Coated Sharpening Cone (installed)
- Wrench with Diamond Smoother
- Plastic Test Stick
- 110v US - Prong Power Cord
- InstRenew Training DVD



InstRenew Sharpening Stone

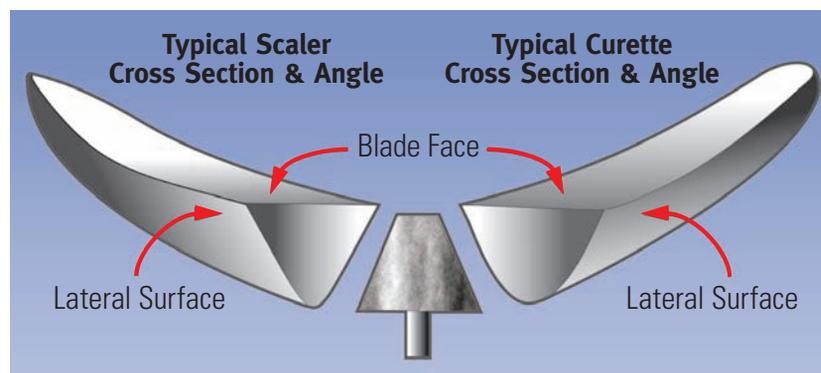

With InstRenew you focus on sharpening the blades and forget about positioning the shanks. Every Brand and Every Pattern of Scalers and Curettes have one thing in common. The angle between the blade face and the lateral surface, that creates the sharp edge, is always the same.

3-Step Method



Position the Blade

The Blade Positioner automatically locks EVERY instrument into the correct position for sharpening by holding the blade face horizontal.



Grasp the Tip

The Tip Clamp allows you to easily grasp the instrument on any straight portion of the shank and maintain the correct position while you sharpen.



Sharpen the Blade

The Sharpening Cone is cut at the correct angle for EVERY brand and EVERY pattern of scalers and curettes. Simply trace each cutting edge over the sharpening cone.

INSTRENEW® SHARPENING ASSISTANT

InstRenew Accessories



Plastic Test Stick
Non-Autoclavable (4" x 1/2")

SSTS



InstRenew Wrench

SSM350



Replacement Sharpening
Cones

Diamond Coated Replacement Cones

SS310 (Fine)



InstRenew Training DVD

SSM310A



Power Cords

For all countries



SSM360
US 2-pole, 2-wire, 110V



SSM370
Australian 2-pole, 3-wire, 240V



SSM375
Cont'l Europe 2-prong, 2-wire 220V



SSM380
UK 2-pole, 3-wire, 220V



Tip Clamp Replacement

Easy to install

SS351

MANUAL SHARPENING SUPPLIES

Test Sticks



Non-Autoclavable Test Stick

4" x 1/2" (10.6cm x 1.3cm).

SSTS




Autoclavable Test Stick

4" x 1/4" (10.6cm x 0.6cm).

SSTSA

Sharpening Stones

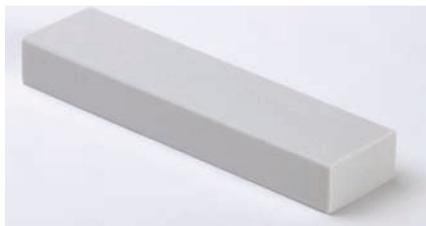

Hard Arkansas Stone #2

Rectangular 4" x 1" x 1/2" (10.6cm x 2.5cm x 1.3cm).

SS2



Hard Arkansas Stone #4

Round 4" x 1/4" (10.6cm x 0.6cm).

SS4



Hard Arkansas Stone #1 Slip Shape

4" x 2-1/2" x 1/8" (10.6cm x 6.4cm x 0.3cm).

SS1



Hard Arkansas Stone #3

Triangular 4" x 3/8" (10.6cm x 1cm).

SS3



India Medium Stone #5 Slip Shape

4-1/2" x 2" x 3/8" (11.4cm x 5 cm x 1cm).

SS5



Ultra-Hard Arkansas Stone #22 Mini Slip Shape

2" x 7/8" x 3/16" (5cm x 2.2cm x 0.5cm).

SS22



Medium Grit Ceramic Stone #6

Flat 3 x 1-1/4" x 1/4" (7.6cm x 3.2cm x 0.6cm).

SS6



Honing Oil For use with manual sharpening stones 1 Oz. (29.5ml) Bottle.

SSHO

Nordent's Professional Sharpening Service

Nordent provides a Professional Sharpening Service that will supplement your in-house instrument maintenance program by reestablishing the original blade angles and factory sharpness. Using Nordent's sharpening service on a regular schedule will make it faster and easier for you to maintain blade sharpness with your in-house sharpening program.

Nordent's craftsmen will inspect and restore the sharpness of ANY BRAND and ANY TYPE of dental instruments within 48 hours of receiving your order. Feel free to contact us if you have any questions, or need instructions on using our service.

To download a PDF which contains our current sharpening price list and order form, please go to www.nordent.com