


# GRADIA® CORE KIT


DUAL-CURED RADIOPAQUE COMPOSITE  
FOR CORE BUILD-UP & POST CEMENTATION

UNIVERSAL

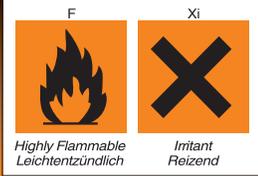


## CARTRIDGE TECHNIQUE GUIDE

- Use GRADIA CORE DISPENSER GUN, available separately, to dispense the material.
- Ensure the logo "THIS SIDE UP" is facing upward when inserting the cartridge in the dispenser.

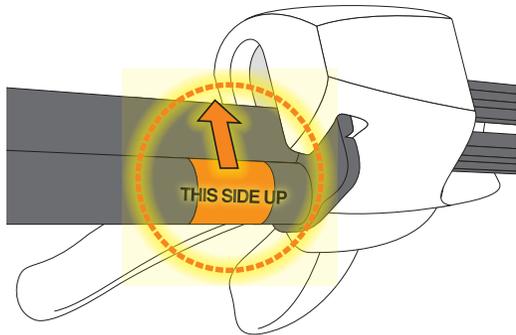

- Rotate the cap 1/4-turn counterclockwise, tilt the cap downward and peel it away from the cartridge.

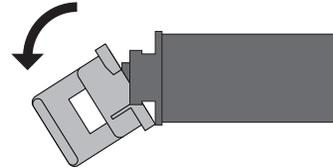

- Store the cartridge, Self-Etching Bond Liquid A and B in refrigerator when not in use.



GC

MANUFACTURED by  
GC CORPORATION  
76-1 HASUNUMA-CHO, ITABASHI-KU,  
TOKYO, JAPAN  
DISTRIBUTED by  
GC CORPORATION  
TOKYO, JAPAN

GC EUROPE N.V.  
B-3001 LEUVEN, BELGIUM  
GC AMERICA INC.  
ALSIP, IL 60803 U.S.A.  
GC ASIA DENTAL PTE. LTD.  
SINGAPORE 508724

MADE IN JAPAN