CE CERAMAGE® Finishing & Polishing Kit

ENGLISH Instructions for Use

This assortment is recommended for systematic finishing and polishing of highly ceramic filled light curing composite materials such as **CERAMAGE**. It includes one **Robot Carbide Cutter** (Fissure Bur), three **Dura-Green Stones**, three **CompoMaster Coarse** (yellow polishers) and three **CompoMaster** (white & yellow bands). First, use **Robot Carbide Cutter** for finishing of the interproximal and occlusal anatomical details and fissures. Then use **Dura-Green Stones** for grinding and contouring the composite surface. Next use **CompoMaster Coarse** for finishing and polishing and **CompoMaster** for super polishing. **CompoMaster Coarse** and **CompoMaster** are durable and filled with a high density of pure diamond particles. They are designed for quick finishing and polishing of high ceramic filled composite restorations to provide a smooth surface. All polishers are autoclavable for reuse.

Also, by using **DURA-POLISH** and **DURA-POLISH DIA** polishing wax sold separately, higher shiny surface gloss can be achieved.

	Recommended min ⁻¹	Maximum min ⁻¹	Recommended pressure
Robot Carbide Cutter (Fissure Bur) (SH97-104-010 / PN 0660)	5,000 – 10,000	50,000	0.5 N
Dura-Green Stones	5,000 – 20,000	30,000	2 N
CompoMaster / CompoMaster Coarse	5,000 – 15,000	20,000	0.5 N

Note

When using polishers intra-orally, operate each polisher under dry conditions with intermittent strokes and supply compressed air to avoid overheating of the composite and to protect the pulp.

Caution

- Operate each instrument up to maximum permissible speed only.
- Insert the shank completely into the chuck of handpiece.
- Rotate each instrument prior to use to check safety and smooth operation.
- Operate at recommended pressure only.
- Do not use at an acute angle.
- Wear eye-protecting glasses when operating.
- Use these products for indicated purpose only.

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SHOFU INC., 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605 - 0983, Japan.
SHOFU INC., Singapore Branch Bldg. 215 Henderson Road # 03-01, Singapore 159554.
SHOFU INC., Shanghai Rep. Office, 1006 Room, Hi-Tech Building, 900# Yishan Road, Caohejing Development Park, Shanghai 200233, China
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, CA 92069, USA.
SHOFU DENTAL PRODUCTS LTD. Duke's Factory, Chiddingstone Causeway, Tonbridge, Kent, TN11 8 JU, UK.
SHOFU DENTAL GmbH, Am Brüll 17, 40878 Ratingen, Germany, EU REPRESENTATIVE.