



# Harvest Speed Polish®

Time is Precious

Zirconia and Lisi Polish

Harvest Dental Products, LLC  
905 Columbia St.  
Brea, CA 92821 USA  
Tel: 1+714-674-7400  
Fax: 1+714-674-7402  
Em: [hello@harvestdental.com](mailto:hello@harvestdental.com)  
Web: [www.harvestdental.com](http://www.harvestdental.com)

## INSTRUCTIONS FOR USE – ENGLISH

### Intended Use:

Made from scratch in small batches by Harvest Dental, Speed Polish® is a polishing compound crafted from a diamond/alumina base specially developed for polishing ZrO<sub>2</sub> and LiSi<sub>2</sub> ceramics. Reduced dust formation and time saving polishing are only two of the outstanding characteristics of Speed Polish.

### Working Instructions:

Lightly apply Speed Polish® to a #9 Standard stiff Bristle Brush at 5,000 RPM. The rotating motion will infiltrate the polish into the bristles. Next, apply Speed Polish® at a low speed and slowly increase RPM as you move the bristle brush over the surface of your restoration with medium pressure for 4-6 seconds, creating a high shine. Remove excess polish with a soft brush, cloth or rinse with water. Speed Polish® is water soluble, reduces wear on opposing dentition and rivals glazed restorations.