GERÄTEDOKUMENTATION

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Transformator

DOC 85985 / 02

- de Deutsch
- en English
- fr Français
- es Español
- it Italiano
- ru Русский

Transformator
33600241

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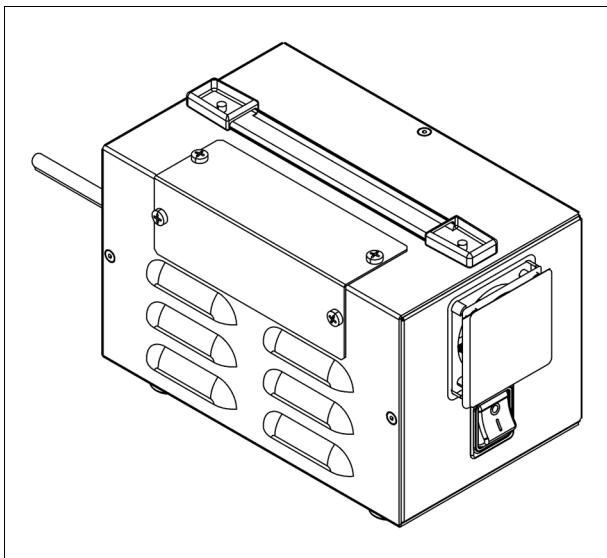
Transformator

de Deutsch

Transformator

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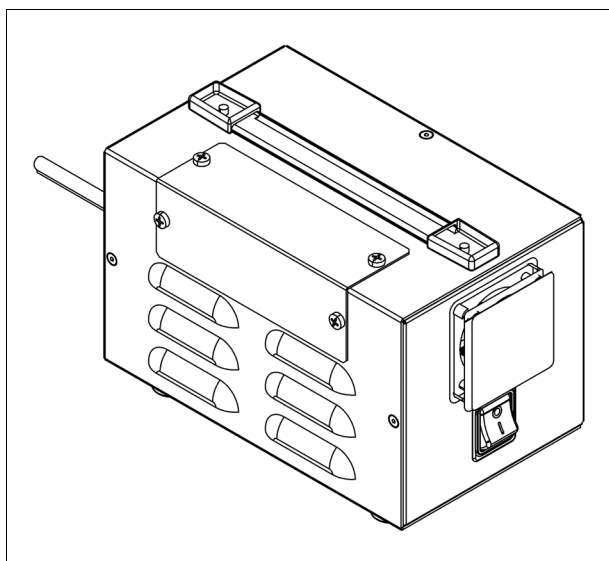
Translation of the original instructions

Transformer

en English

Transformer

This unit documentation is part of the unit and must be enclosed when selling or passing on the unit.

- The unit has been designed solely for use in dental laboratories and comparable establishments for research, commercial and training purposes.
- The operating instructions must be read and understood before the device is used. This particularly applies to the **Safety information**. The warranty is void in cases of damage which is caused by non-observance of the operating instructions. We will also not accept liability for any resulting consequential damages.
- Symbols used

This symbol indicates very important information. Failure to comply with it may result in personal injury.

This symbol warns about hot parts.

This symbol warns of electrical hazards.

This symbol indicates that the device must be disconnected from the power supply and secured against being switched on again (Germany: see the safety regulations for electrical systems).

This symbol indicates that the action may only be carried out by BEGO customer service staff or by persons authorised by them.

Table of contents

General information.....	15
Safety instructions.....	17
Unit description	19
Scope of delivery.....	19
Technical data	19
Installation	20
Operation	21
Care and maintenance.....	21
Elimination of malfunctions	22
Signs, labels and stickers.....	22
Disposal	23
Wiring diagram	Appendix
Certificate of Conformity.....	Appendix

General Information

Importance of the operating Instructions

These operating instructions cover all the information that is necessary according to the relevant regulations for the safe operation of the device described herein.

The operating instructions are a part of the machine. The operating instructions should therefore

- always be kept at hand near the device until the device is disposed of,
- and should be passed on with the device when it is purchased, sold, or lent.

Contact the manufacturer if you do not understand something in the operating instructions.

We welcome any suggestions or criticism. Please feel free to notify us. Your effort will help us make the operating instructions more user-friendly and respond more effectively to your wishes and needs.

Target group

This document is directed toward everyone who works with this device or performs service tasks that are described in this document.

Contact information

BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG
Wilhelm-Herbst-Str. 1
28359 Bremen • Germany
Telephone: +49 421 2028-0
Service-Hotline Tel.: +49 421 2028-270
www.bego.com

© Copyright by BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG

This user guide is protected by copyright. All rights reserved.

No part of this document may be reproduced without the express written permission of BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG!

Conventions

These operating instructions contain references to residual hazards, important user tips and handling instructions that are identified with the following symbols and words.

DANGER

This reference identifies hazards that can cause serious bodily injury or death if the relevant hazard instructions are not followed or not followed properly.

WARNING

This reference identifies hazards that can cause bodily injury or property damage if the relevant hazard instructions are not followed or not followed properly.

CAUTION

This reference identifies only those hazards that are potential damaging to property and the environment.

NOTE

This symbol identifies user tips and particularly useful information. It helps you optimally exploit all the functions of your machine.

1. Sequential actions are described in sequentially numbered paragraphs.
- Cross-references are identified with this symbol.

Warranty and Liability

Our general terms and conditions of sale and delivery apply. These are made available to the customer on completion of the contract, at the latest. Warranty and liability claims in the event of physical injury or damage to property are invalid if they are caused by one or more of the following:

- Improper use of the device
- Improper installation, putting into operation, operating and maintenance of the device;
- Operating the device with safety and protective facilities that are defective, improperly installed or inoperative
- Failure to heed the information in this manual concerning the transport, storage, installation, putting into operation, operation and service and maintenance of this device;
- Unauthorized structural modifications to this device;
- Inadequate monitoring/checking of parts of the device that are subject to wear;
- Improperly performed repairs;
- Catastrophes caused by alien elements or force majeure.

Safety instructions

The Transformer is only intended for connection to BEGO casting equipment. Any other use is considered to be improper. We shall not be liable for any damage resulting from improper use.

Incorrect use is, for example, the use with equipment other than that which is stated. Misuse can cause hazards such as short circuit, fire and electric shock!

The safety instructions must be followed in order to prevent personal injury and damage to the machine. We shall not be liable for damage due to failure to comply with the safety instructions.

General Safety Instructions

Place of use

The unit has been designed solely for use in dental laboratories and comparable establishments for research, commercial and training purposes.

The unit may only be installed on an adequately stable surface.

The unit must only be used in dry rooms.

Check the machine and leads for damage regularly. The machine must not be operated if it has any defects that may result in risks for the employees or third persons.

Protect the leads against heat, oil and sharp edges. The leads must not be used to carry the machine or to pull out the mains plug.

Operator and operating personnel

Wear the protective clothing prescribed for use in dental laboratories and observe the rules of conduct.

The unit may only be operated by staff members who are familiar with and follow the instructions of this operating manual.

The operating manual must be easily accessible when the machine is operated. Observe the national accident prevention regulations in addition to this operating manual.

Repairs

Repairs may be carried out solely by BEGO customer service or by persons authorised by BEGO customer service.

Metallic housings must be grounded properly so as to prevent them from carrying current. Otherwise they are extremely dangerous because the housings may be live due to damage to the machine! Since a check to ensure a de-energized state is mandatory after every intervention in the machine (Germany: check in accordance with DIN VDE 0701-1), only qualified electricians may open the machine!

Handling the machine

Prior to the electrical connection check whether the specifications on the rating plate correspond to the power supply network. In the case of uncertainty, consult an electrician.

The unit is categorised under protection class 1 and may only be connected to properly earthed power sources (earth contact sockets).

Modifications of the machine are not permitted.

Always keep signs and stickers such that they are easy to read. They must not be removed.

Care and maintenance

The unit has been designed for a service life of 10 years from the date of manufacture. No liability is accepted for damages arising from operation of the unit after this period.

Switch off the machine and pull out the mains plug before performing any care, cleaning or maintenance work.

Only use dry or slightly moist cloths for cleaning. Do not spray the machine with water or immerse it in water.

Special safety instructions regarding this machine

⚠ DANGER

Warning of electric shock! Danger to life!

Installation may only be carried out by BEGO customer service staff or persons authorised by them (trained electricians)!

- Incorrect installation may cause a hazard to life due to electric shock!
- The connected casting equipment may be destroyed by incorrect installation.
- Do not allow liquids to enter the device. Danger to life due to electric shock!
Select a protected installation location.

⚠ CAUTION

Property damage!

- When laying cables, ensure that they are protected from hot components (e.g., moulds).
- The unit must only be used in dry rooms.
- The device requires cooling:
 - Maintain a distance of at least 10 cm from adjacent equipment.
 - Do not cover the cooling slits.
 - Maximum ambient temperature 40 °C.

⚠ WARNING

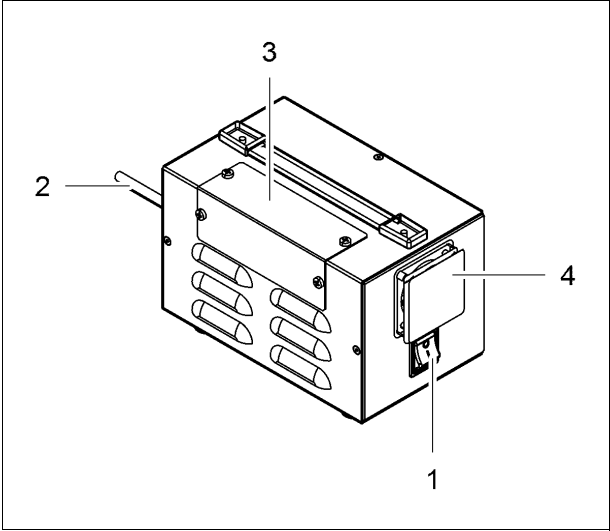
Burn hazard from hot parts!

- The device becomes hot during operation. Caution when touching.
- Carry out maintenance and cleaning work before working or only after the machine has cooled down.

Unit description

Transformer

The Transformer is only intended for connection to BEGO casting equipment. It is required if there is no 230 V, 50/60 Hz power supply at the installation site. The Transformer converts alternating current in the range 200 to 240 V, 50/60 Hz to 230 V, 50/60 Hz.

- 1 Main switch with thermal protection
- 2 Connection cable (200...240 V, 50/60 Hz)
- 3 Cover
- 4 Connection socket (230 V, 50/60 Hz)

Scope of delivery

Transformer

200...240 VAC, 50/60 Hz REF 17800

with:

- Operating instructions

Technical data

Height.....	145 mm	Power output	3680 VA (40 % Duty Cycle)
Width	127 mm	Protection class	I
Depth	245 mm	Protection type	IP 23
Weight.....	5.6 kg	Ambient temperature (max.)	40 °C
Input voltage	200/210/220/230/240 VAC		
Input current (max.).....	19.4 A		
Output voltage.....	230 VAC		
Output current (max.).....	16 A		

Installation

⚠ DANGER

Warning of electric shock! Danger to life!

Installation may only be carried out by BEGO customer service staff or persons authorised by them (trained electricians)!

- Incorrect installation may cause a hazard to life due to electric shock!
- The connected casting equipment may be destroyed by incorrect installation.
- Do not allow liquids to enter the device. Danger to life due to electric shock!
Select a protected installation location.

⚠ CAUTION

Property damage!

- When laying cables, ensure that they are protected from hot components (e.g., moulds).
- The unit must only be used in dry rooms.
- The device requires cooling:
 - Maintain a distance of at least 10 cm from adjacent equipment.
 - Do not cover the cooling slits.
 - Maximum ambient temperature 40 °C.

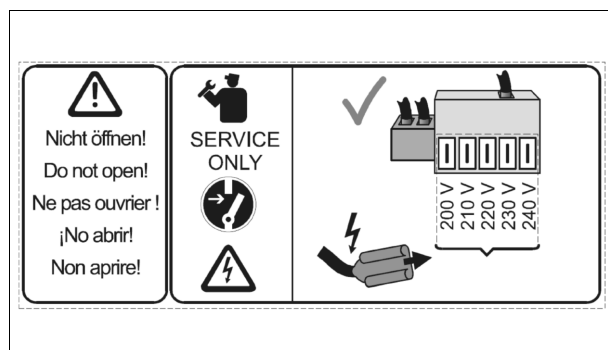
Information for electricians

⚠ WARNING

Danger due to electric shock

- The machine is subject to the provisions of class of protection I and may only be connected to properly grounded power sources.
- When laying cables, ensure that they are protected from hot components (e.g., moulds).

⚠ WARNING

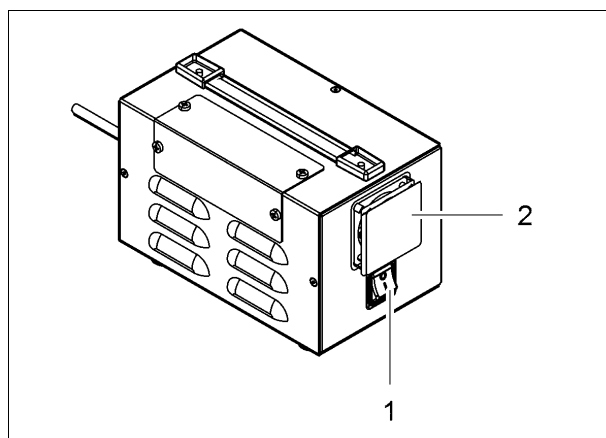
Danger due to electric shock

Disconnect the device from the power supply and secure against switching on.

1. Open the cover and plug the cable connectors to the relevant terminals.
Do not reconnect any other cables.
2. Close the cover.

Operation

1. Connect the casting machine to plug socket (2).
2. Switch on the main switch (1).
The main switch illuminates when the device is switched on.

⚠ WARNING

Burn hazard from hot parts!

The device becomes hot during operation. Caution when touching.

Care and maintenance

⚠ DANGER

Warning of electric shock! Danger to life!

Switch off the machine and pull out the mains plug before performing any care or maintenance work.

In the case of power connection without mains plug: remove fuses, secure machine so it cannot be switched on again and check that it is de-energized.

⚠ WARNING

Burn hazard from hot parts!


Carry out maintenance and cleaning work before working or only after the machine has cooled down.

Only use dry or slightly moist cloths for cleaning. Do not spray the machine with water or immerse it in water.

Elimination of malfunctions

Malfunction	Cause	Elimination
The main switch does not illuminate when the device is switched on:		
a) Switch jumps to OFF	The thermal protection has triggered due to overload	Switch the main switch on again after about 10 min.
b) Switch remains switched on	No power supply	Check the building mains and fuses.

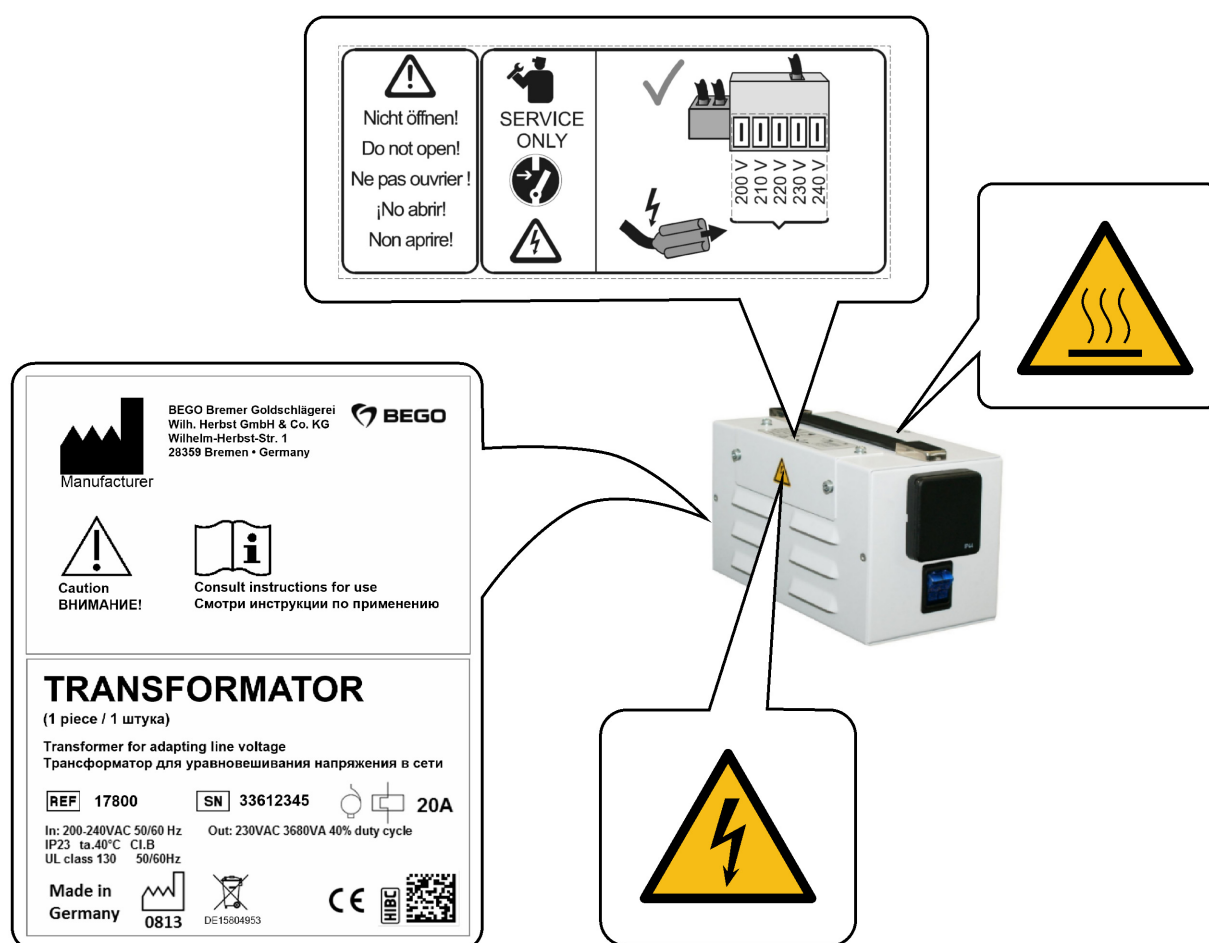Signs, labels and stickers

Disposal

Instructions for disposing of the device

(Applicable only within the European Union)

The adjacent symbol on the ID plate of the BEGO device indicates that the device, in accordance with the European directive on waste electrical and electronic equipment, may not be disposed of as normal domestic waste.

As a customer, you contribute to the protection of the environment when you dispose of the device correctly.

Disposal in Germany

BEGO offers you a disposal solution for all BEGO devices sold and put into operation in Germany subsequent to August 13, 2005. Please contact us when it is time to dispose of the device.

Disposal in other countries of the European Union

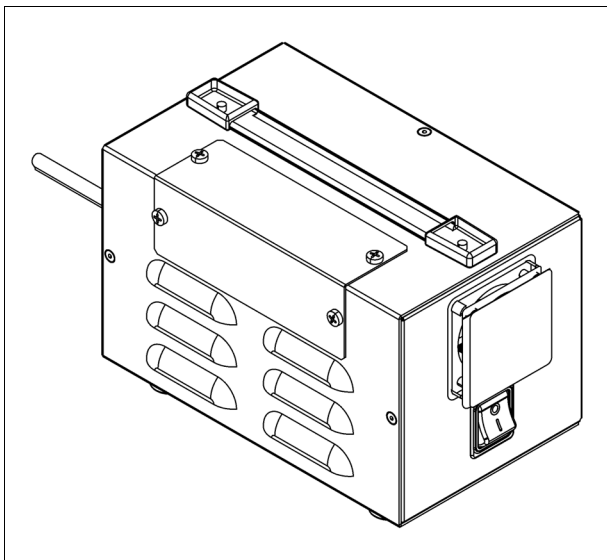
Please contact the company from which you purchased the BEGO device when it is time to dispose of it. They will provide you with information concerning correct disposal in your region.

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