

# MonoCem<sup>®</sup>

Self-Adhesive Resin Cement



Visit [www.shofu.com](http://www.shofu.com) | Call 800.827.4638

# CASE STUDY



**One:** Clean, isolate and moisten prepared tooth.



**Two:** Inject MonoCem into crown and seat on tooth.



**Three:** Apply occlusal pressure.



**Four:** Light cure, maintaining occlusal pressure.



**Five:** Remove excess and polish margins of the crown.

Photos courtesy of George Freedman DDS, FAACD,

## MonoCem®

- Bonds to all substrates with high retention values
- Moisture tolerant with 2 second tack cure for easy clean-up
- Eliminates sensitivity and has high fluoride release
- 100% polymerization/dual cure: 3 minute auto cure; 20 second light cure
- Command set, curing begins at placement
- Very low film thickness (<12µm)

Visit [www.shofu.com](http://www.shofu.com) | Call **800.827.4638**



MonoCem self adhesive resin cement is a very simple to use and very effective cement. It handles well with excellent flow properties and clean up is easy. MonoCem exhibits all of the ideal properties of a cement used for luting adhesive ceramic restorations.

**Eugene L. Antenucci, DDS**  
Huntington, NY

I use MonoCem because I only use the best for my patients.

**Robert Girschek, DMD**  
Waltham, MA



### Order Information

- Part Number 3208A
- (2) 3.5 Translucent syringes
- Part Number 3209A
- (2) 3.5 Bleach White syringes
- 15 Applicator tips
- 2 OneGloss® PS Midi-Points
- Directions for Use



Shofu Dental Corporation  
1225 Stone Drive  
San Marcos, CA  
92078-4059, USA

SN03-1018

# Self-Adhesive Resin Cement