4

INSTRUCTIONS FOR USE (IFU)

MI Paste Plus®

Last revised: 04/2018

5540418

MI Paste Plus®

DENTIFRICE CRÈME WITH BIO-AVAILABLE CALCIUM, PHOSPHATE AND FLUORIDE

For use only by a dentist or physician in the recommended indications.

DESCRIPTION

MI Paste Plus is a water based dentifrice crème containing sodium fluoride and RECALDENT™ (CPP-ACP: Casein Phosphopeptide-Amorphous Calcium Phosphate). The level of fluoride is 0.20% w/w, (900ppm). When CPP-ACP is applied in the oral environment, it will bind to bio-film, plaque, bacteria, hydroxyapatite and soft tissue localizing bio-available calcium, phosphate and fluoride.

IMPORTANT:

Saliva will enhance the effectiveness of CPP-ACP and the flavor will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

RECOMMENDED INDICATIONS

To be used for cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. Secondly, MI Paste Plus can be used for the management of tooth sensitivity, post-scaling, root planing and bleaching.

CONTRAINDICATIONS

RECALDENT™ (CPP-ACP) is derived from milk casein. Do not use on patients with a milk protein or hydroxybenzoates allergy.

DIRECTIONS FOR USE

For children 12 and under, consult a dentist or physician.

Adults and children over 12:

In-office application:

I. Custom tray application

1. Before use rinse the custom tray thoroughly under running water.
2. Extrude a generous layer of MI Paste Plus into the tray and apply to the upper and/or lower teeth.
3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
4. Remove the tray.
5. Instruct the patient to use the tongue to spread the remaining MI Paste Plus throughout the mouth. Instruct the patient to retain for as long as possible (another 1- 2 minutes) avoiding expectoration and delaying swallowing. The longer MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
6. Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
7. Any residual MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

II. Non tray application:

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.
2. Apply MI Paste Plus to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush.
3. Then use your tongue to spread MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (minimum of 3 minutes). Avoid expectoration (spitting out) and delay swallowing.
4. Ask the patient to expectorate thoroughly and avoid rinsing. Any MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

At-home application:

III. Day time application after tooth brushing as recommended by a dentist or physician

1. Apply a sufficient amount of MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with MI Paste Plus.
2. Then use your tongue to spread MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (minimum of 3 minutes). Avoid expectoration (spitting out) and delay swallowing. The longer MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
3. Expectorate thoroughly and if possible avoid rinsing. Any MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dentist or physician

Note:

The overnight use is especially recommended for high-risk adult patients.

1. Apply a sufficient amount of MI Paste Plus to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with MI Paste Plus.
2. Then use your tongue to spread MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (minimum of 3 minutes). Avoid expectoration (spitting out) and delay swallowing.
3. Expectorate thoroughly and avoid rinsing. Any MI Paste Plus remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

FLAVORS

Strawberry (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)

STORAGE

Recommended for optimal performance; store in a cool, dry low humidity area away from direct sunlight (8-25°C; 46.4-77.0°F).

PACKAGES

40g (35mL) Tube

NOTE

MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia. "RECALDENT™" is a registered license of RECALDENT Pty Ltd. CPP-ACP Technology has related patents pending in Australia, NZ, Europe, Canada and the USA.

CAUTION

1. Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
2. Keep out of reach of children under 6 years of age
3. If you accidentally swallow more than used for brushing, get medical help or contact a poison control center right away.
4. In case of contact with eyes, flush immediately with water and seek medical attention.
5. In case of spillage on clothes, wash off with water.
6. CAUTION: RECALDENT™ is derived from milk casein. Do not use on patients with a milk protein or hydroxybenzoates allergy.
7. In case of allergic reaction, stop use, rinse mouth with water and seek medical advice.

Last Revised: 04/2018

MANUFACTURED by
GC AMERICA INC.
ALZIP, IL 60803 U.S.A.

DISTRIBUTED by
GC AMERICA INC.
ALZIP, IL 60803 U.S.A.

PRINTED IN THE USA

Prior to use, carefully read
the instructions for use

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