



FIND SUCCESS IN OVER 33 MILLION RESTORATIONS

GC Fuji II LC®

Resin-Modified Glass Ionomer Restorative

#1 Top-Selling Glass Ionomer Restorative in the US

- Excellent translucency for outstanding esthetics restorations
- High rechargeable fluoride release with virtually no recurrent decay
- Strong chemical bond to tooth resulting in exceptional marginal seal
- 1.5 times the radiopacity of enamel (315% AI)

Townie Choice Award® Winner in Restorative Dentistry
– Glass Ionomers category for last three years.





Outstanding Physical Properties & Handling

Featuring an outstanding chemical bond to tooth structure, GC Fuji II LC®'s high compressive, tensile and flexural strengths assure longevity and integrity. Its consistency makes it easy to place and contour. And, like all GC glass ionomers, it works perfectly in a moist field.

Excellent Translucency!


GC Fuji II LC®

Vitremer™†


Photac™ Fil Quick†

Indication						
Primary teeth	✓	✓	✓	✓	✓	✓
Permanent teeth	✓	✓	✓	✓	✓	✓


Closed Sandwich Technique Using GC Fuji II LC®

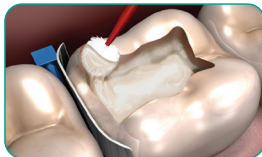


Class II preparation with shallow supragingival box. CAVITY CONDITIONER (GC) is recommended to remove the smear layer.

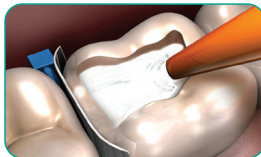


Placement of GC Fuji II LC® occlusally.

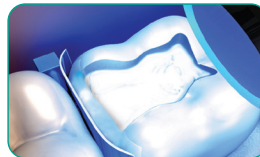

Light-curing of GC Fuji II LC®.

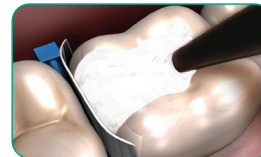


Apply G-Premio BOND™, then place G-aenial Sculpt® composite, followed by light-curing.

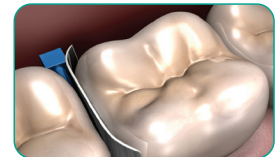


Final, esthetic glass ionomer and composite sandwich technique restoration.

Class V Restorations using GC Fuji II LC®

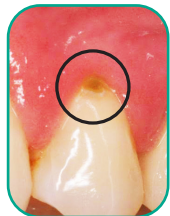

Before

Photos courtesy of Dr. Robert Margeas, Des Moines, Iowa

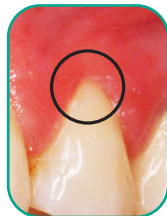

After



Before

Photos courtesy of Dr. John Cornisi, Ithaca, New York



After



GC Fuji II LC® Capsule Refills (US)

Contains: 48 capsules.

425001 A1 425002 A2 425003 A3 425004 A3.5 425005 A4
425006 B2 425007 B3 425008 B4 425009 C2 425010 C4

Contains: 24 capsules.

425020 Dark Shades 425015 Light Shades

GC Fuji II LC® 15g Powder Refills

000195 A1 000196 A2 000197 A3 000198 A3.5 000199 A4
000200 B2 000201 B3 000202 B4 000203 C2 000204 C4

GC Fuji II LC® Capsule Refills (Canada)

Contains: 48 capsules.

442001 A1 442002 A2 442003 A3 442004 A3.5 442005 A4
442006 B2 442007 B3 442008 B4 442009 C2 442010 C4

Contains: 24 capsules.

442020 Dark Shades 442015 Light Shades

000164 GC Fuji II LC® 6.8mL Liquid Refill

000110 CAVITY CONDITIONER (GC)

Contains: 5.7mL liquid.

GC Fuji II LC® 1:1 Kit

Contains: 15g powder, 6.8mL liquid, mixing pad and measuring scoop.

000184 A1 000185 A2 000186 A3 000187 A3.5
000188 A4 000189 B2 000190 B3 000191 B4
000192 C2 000193 C4

013764 NEW GC CAPSULE APPLIER V

Contains: One capsule applier.

