

# Cleaning, Finishing and Superpolishing



## OneGloss and SuperBuff

### OneGloss – Finishing and polishing with one instrument

OneGloss provides the ideal finish and polish to all composite restorations simply by altering the contact pressure. The polishers are equally excellent for removing excess resin or cement as well as surface stains from teeth without marring the restoration or the enamel.

### Super-Snap SuperBuff - High lustre for all composite restorations

SuperBuff is a paste impregnated felt disc designed for superpolishing of direct and indirect composite restorations. Easily activated by water, the disc develops a creamy foam that perfectly polishes all surfaces.



Proven Products for Better Dentistry

# The solution for all composite restorations

## OneGloss – Finishing and polishing

**OneGloss** instruments are made of highly concentrated aluminium oxide with silicone as binder.

**OneGloss** is a one-step finisher and polisher. By simply altering the contact pressure, you can go from finishing to polishing.

- pressure of approx. 1.0 Newton → Finishing
- pressure of approx. 0.3 Newton → Polishing



Shape	Cup	Midipoint	IC	Minipoint	Mandrel
ISO-No.	060	060	100	030	
PN	0181	0182	0183	0186	0435
Quantity	50 pcs.	50 pcs.	50 pcs.	12 pcs.	6 pcs.

**OneGloss** is available in the following shapes:

- Cup for cusp, incisal and occlusal
- Midipoint for occlusal, labial, lingual and buccal
- IC for labial, lingual, buccal, interproximal
- Minipoint for fissures

### OneGloss Kit

60 unmounted polishers (20 each Cup, Midipoint, IC) and 3 CA mandrels  
PN 0180

## SuperBuff – Superpolishing



Simply moistened before use, the aluminium oxide impregnated **SuperBuff** felt disc is activated and produces a creamy paste. The front and the flat lateral edge of the disposable instrument can be used on multiple surfaces. The unmounted disc snaps on to a Super-Snap mandrel.

- impregnated discs eliminate mess
- faster than polishing with paste
- polishes all composites
- activated by water or saliva

### Super-Snap SuperBuff Kit

25 polishers and 2 mandrels, CA shank  
PN 0535

**Refill** with 50 pieces  
PN 0536

**FG mandrels**, box with 6 pieces  
PN 0438

**CA mandrels**, box with 6 pieces  
PN 0439



Super-Snap SuperBuff Kit  
PN 0535



OneGloss Kit  
PN 0180



### SHOFU DENTAL GMBH

Am Brüll 17 · 40878 Ratingen / Germany  
Phone: +49 (0) 21 02 / 86 64-0 · Fax: +49 (0) 21 02 / 86 64-64  
E-mail: info@shofu.de · www.shofu.de