

Cleaning, Finishing and Super Polishing



OneGloss® and SuperBuff®

OneGloss – The One Step Finisher and Polisher

OneGloss instruments provide the ideal finish and polish to all composite restorations in one step. The polishers are excellent for removal of excess resin or cement without marring delicate inlays as well as removal of surface stains from teeth without marring the enamel.

SuperBuff – The Super Polisher for All Composite Restorations

SuperBuff is a paste-impregnated felt disc designed for super polishing of direct and indirect composite restorations without the use of added polishing pastes. Easily activated by water or saliva, the disc perfectly polishes all surfaces.



Proven Products for Better Dentistry

The Solution for All Composite Restorations

OneGloss® – Two-in-One

OneGloss instruments are made of highly concentrated aluminium oxide with silicone as binder.

OneGloss is a one-step finisher and polisher.

By simply altering the contact pressure, you can go from finishing to polishing.

– pressure of approx. 1.0 Newton → Finishing

– pressure of approx. 0.3 Newton → Polishing

OneGloss is available in the following shapes:


- Cup for cusp, incisal and occlusal
- Midi-Point for occlusal, labial, lingual and buccal
- IC for labial, lingual, buccal, interproximal
- Mini-Point for fissures



| Shape | Cup | Midi-Point | IC | Mini-Point | Mandrel |
|---------|------|------------|------|------------|---------|
| ISO-No. | 060 | 060 | 100 | 030 | |
| PN | 0181 | 0182 | 0183 | 0186 | 0435 |

The **OneGloss Kit** contains 20 pcs each of Cup, Midi-Point and IC as well as 3 CA mandrels.
PN 0180

SuperBuff® – Final Polish for Composite Restorations

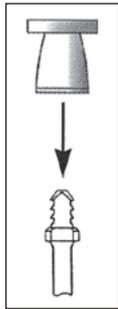

Simply moistened with water or saliva before use, the aluminium oxide impregnated **SuperBuff** felt disc is activated and produces a creamy paste.

The front and the side of the disposable instrument can be used on multiple surfaces, even pits and fissures. The unmounted disc snaps on to a Super-Snap mandrel.

- **impregnated discs eliminate mess**
- **faster than polishing with paste**
- **polishes all composites**
- **activated by water or saliva**

SuperBuff Set: 25 discs and 2 CA mandrels

SuperBuff Disc refills: 50 pcs

Super-Snap CA mandrel: 6 pcs

PN 0535

PN 0536

PN 0439



SuperBuff® Kit
PN 0535

OneGloss® Kit
PN 0180



SHOFU DENTAL GMBH

Am Brüll 17 · 40878 Ratingen / Germany
Phone: +49 (0) 21 02 / 86 64-0 · Fax: 86 64-64
E-mail: info@shofu.de · www.shofu.de