## EN Overcoming challenges in teeth whitening: tetracycline staining

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63-year-old female patient.

### COMPLAINT

Patient wanted to whiten her teeth without having to resort to veneers.

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### EN CLINICAL/RADIOGRAPHIC EXAM

Teeth showing a grade 2 tetracycline with brown and yellow bands. The most saturated shade of the bands corresponded to the 3R 2.5 of the VITA 3D MASTER shade guide.

### TREATMENT

Due to the etiology of the staining, we opted for supervised at-home whitening with the use of 16% carbamide peroxide (Whiteness Perfect) in custom trays for 3 to 4 hours per day for 2 months. The photographs show that in one month of treatment the shade of 2L 1.5 was reached with much lighter bands. At 2 months, the result was a shade corresponding to OM 2 and the bands became practically unnoticeable.

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4. Instructing the patient to use Whiteness Perfect 16% in the trays for 4 weeks for 3 to 4 hours per day.
5. Follow-up after 4 weeks of treatment, showing an improvement in the shade. Facing the report of the absence of sensitivity, the patient was instructed to continue treatment with Whiteness Perfect 16%, for 3 to 4 hours per day.

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## FGM MATERIALS USED | MATERIALES FGM UTILIZADOS

TAKE-HOME WHITENING GEL  
(CARBAMIDE PEROXIDE)  
**Whiteness Perfect 16%**

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PLATES FOR TRAYS  
**Whiteness Plates for Custom Dental Trays**

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