

OPERATION MANUAL

Master SAW



BesQual

/ Install manual

1. Plug pedal
2. Plug main power
3. Install draw


/ Operating manual

1. Power on
 2. Check laser beam is exhaust on sawing blade
 3. Placing model on plate with laser line
 4. Select operating mode
 - Palm mode : Both safety switches are pushed for run
 - Pedal mode : Run while pedal is pushed.

Must check nothing on(pusing) pedal before changing mode.
 5. Turn on the dust collector
 6. Hold model with two hands tightly and push down plate slowly.
Quick operate may stop the running.
- How to reset
- Palm mode remove palm from safty plate just while
 - Pedal mode lift the foot from pedal and push again
7. Remove dust from dust draw (do not need every time)

/ Disk exchange manual

1. Remove power supply (must disconnet it, for safty)
2. Remove dust draw
3. Stand up the machine from left to upward
4. Hold a disc tightly and turn the screw at anticlockwise with allen wrench, if it released using a finger
5. Remove used sawing disk
6. Placing holding plate(Male) as picture shows. ①
7. Placing sawing disk as picture shows ② if placing oppositem, disk may broken
8. Placing holding plate(Female) as picture shows. ③,④
9. Lock the screw at clockwise, at last holding a disk and using a allen wrench and lock tightly

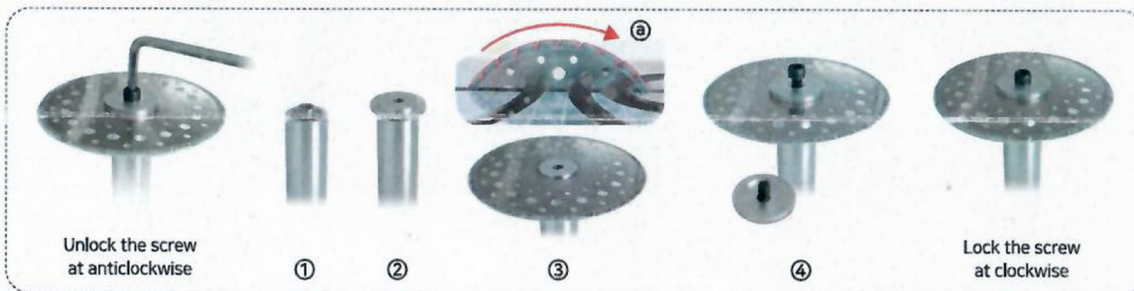

/ Caution

1. Using for cutting palster parts only (if cut plastic; plastic may melt on disc and reduce performance)
2. For safety reasons **recommend palm mode**
3. **Never watching laser** directly
4. Power off after using
5. Keep away from children.


/ Part name

► Front




- ① Model palcing plate
- ② Guide laser
- ③ Safely button for palm mode
- ④ Mode select switch (palm, foot pedal)
- ⑤ LED Light palm mode
- ⑥ LED Light foot pedal
- ⑦ Main power switch
- ⑧ Dust draw

► Back

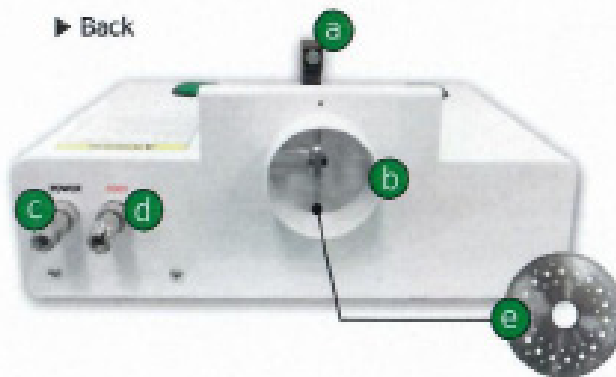

- a Laser
- b Dust collector connecting hole
- c Power Outlet
- d Pedal Outlet
- e Sawing disk


/ LASER beam

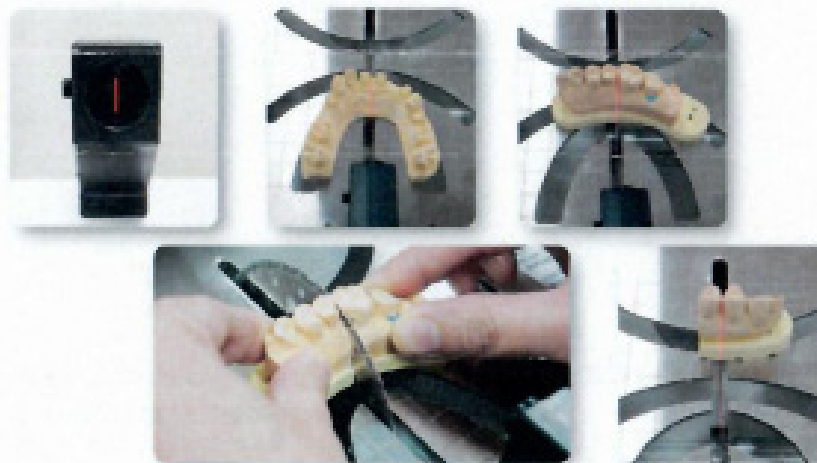

► ACCESSORY



Pedal

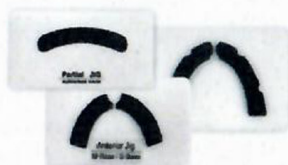

Jig
[Partial, Anterior, Full Arch]



24V Adapter

/ Product Spec

Product name	Master SAW
Size	W340*L235*H130 (mm)
Weight	6 kg
Power	DC 24V . 4A



META DENTAL CORP. +1-800-730-META(6382) New York U.S.A
Website : www.metadentalco.com