
ceramill motion 2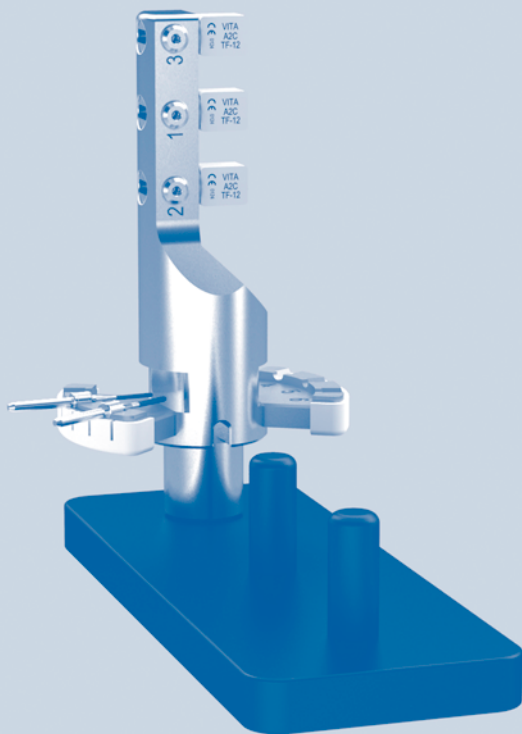

Gebrauchsanweisung Instruction Manual Instructions d'utilisation Istruzioni per l'uso Instrucciones de uso

Deutsch	3 - 7
English	8 - 12
Français	13 - 17
Italiano	18 - 23
Español	24 - 28

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Table of Contents

1	Explanation of Symbols	8
2	Suitable Personnel	9
3	Product description	9
4	Application	9
4.1	Loading the Blank Holder for Glass Ceramic Blanks	9
4.2	Preparing the Ceramill Motion 2	10
4.3	Inserting the Blank Holder for Glass Ceramic Blanks	11
4.4	Starting the Grinding Program	12
5	Warranty/Exclusion of Liability	12
6	Download Infos	12

1 Explanation of Symbols

Warning indications

Warning indications in the text are marked with a colour-backed triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- **NOTE** means that property damage can occur.
- **CAUTION** means that light to fairly serious personal injury can occur.
- **WARNING** means that serious personal injury can occur.
- **DANGER** means that serious personal injury can occur.

Important information

Important information that do not lead to hazards for humans or property damage are marked with the icon aside and are also boxed.

Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
–	Item of a list
▪	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

Tab. 1

2 Suitable Personnel

NOTE:

The product may only be used by trained specialised personnel.

3 Product description

The Ceramill Motion 2 is available as four-axis model (4X) and as five-axis model (5X). These Operating Instructions apply for both models.

The blank holder for glass ceramic blanks is a blank holder for Ceramill Motion 2. It is used to accommodate up to three glass ceramic blanks and the respective Ceramill Motion 2 grinding tools.

A blank holder each is available for the Ceramill Motion 2 5X and 4X:

Ceramill Motion 2	Blank holder
5X	179260
4X	179281

Tab. 2 Art. No. of the blank holders

Delivery Scope

- _ Blank holder for glass ceramic blanks (3-fold)
- _ Storage rack

4 Application

4.1 Loading the Blank Holder for Glass Ceramic Blanks

The blank holder can also be inserted into the Ceramill Motion 2 before loading it (see page 12).

- ▷ Insert the Ceramill Motion 2 standard tools into the blank holder for glass ceramic blanks. Position the tool shank facing toward the outside.
 - Mount the Ceramill Motion 2 grinding point 1.8 at position 1.
 - Mount the Ceramill Motion 2 grinding point 1.4 at position 2.
 - Mount the Ceramill Motion 2 grinding point 1.0 at position 3.

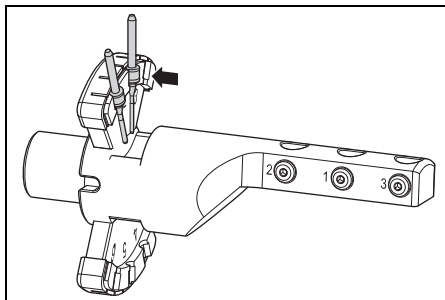
Fig. 1

- ▷ Optionally: Insert the Ceramill Motion 2 sister-grinding tools into the blank holder for glass ceramic blanks. Position the tool shank facing toward the outside.
 - Mount the Ceramill Motion 2 grinding point 1.8 at position 4.
 - Mount the Ceramill Motion 2 grinding point 1.4 at position 5.
 - Mount the Ceramill Motion 2 grinding point 1.0 at position 6.

- Depending on your selection in the Ceramill Match 2, insert up to three blanks into the blank holder for glass ceramic blanks with the groove facing forward, and secure them with the Allen key.

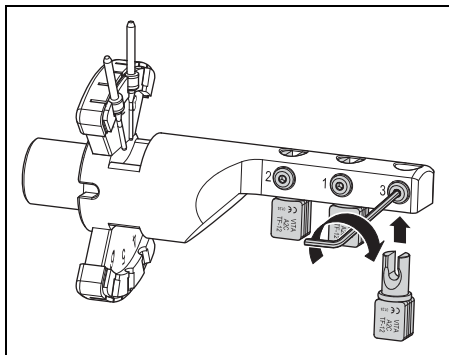
Fig. 2

4.2 Preparing the Ceramill Motion 2

When the Ceramill Motion 2 is set up for dry processing:

- Convert the Ceramill Motion 2 for wet processing (see Ceramill Motion 2 Operating Instructions)
- Disengage lever [3] and pull off the transparent splash guard [1] toward the front.

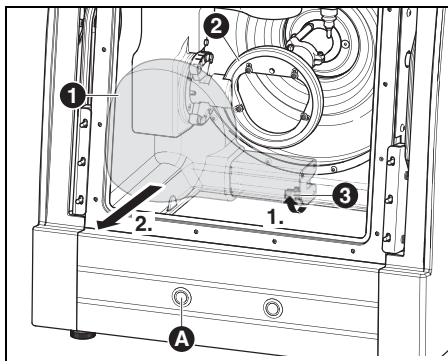
Fig. 3 Five-axis model

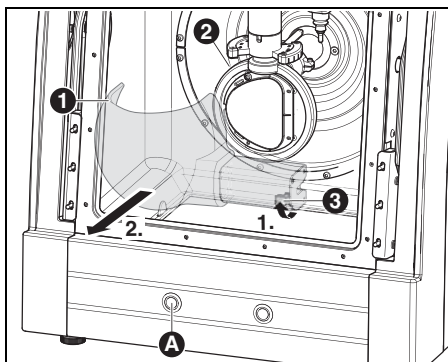
Fig. 4 Four-axis model

Legend to figure 3 and figure 4:

- 1 Splash guard
- 2 Blank holder
- 3 Lever
- A Operation button

To remove the blank holder:

- ▷ With the door closed, press operation button [A] for at least three seconds.

The current tool is placed down and the blank holder [2] returns to the change position.

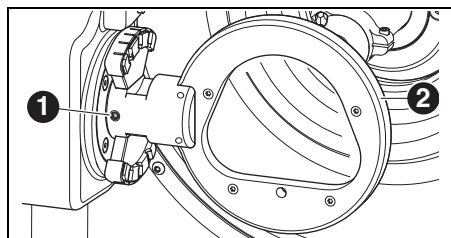
Fig. 5 Five-axis model

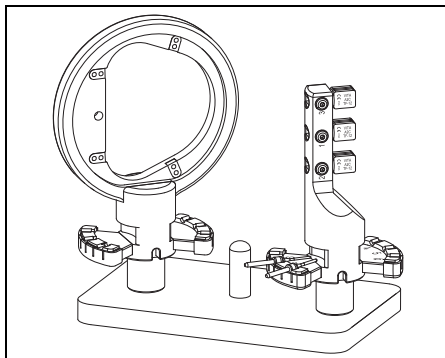
Fig. 7

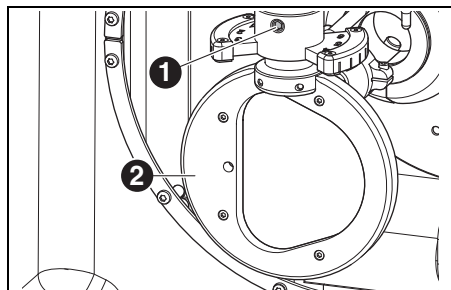
Fig. 6 Four-axis model

Legend to figure 5 and figure 6:

1 Fastening screw for blank holder

2 Blank holder

- ▷ Loosen fastening screw [1].

- ▷ Pull off the blank holder toward the right (five-axis model) resp. toward the bottom (four-axis model).

- ▷ Clean the locating bore and surfaces.

Blank holders not in use can be placed on the storage rack and thus securely stored.

4.3 Inserting the Blank Holder for Glass Ceramic Blanks

- ▷ Check the pin of the blank holder and clean if required.
- ▷ Insert the blank holder with the pin **to the stop** in the locating bore.

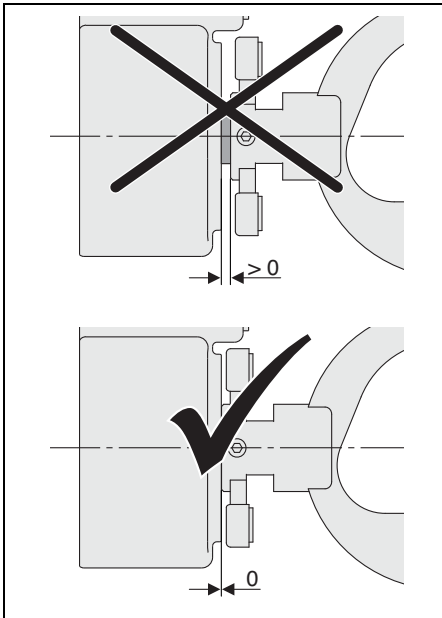
Fig. 8

NOTE:

Improper handling leads to inaccurate working results or to damage of the blank-holder fixture!

- ▷ Tighten fastening screw [1] with a torque of 4 Nm.

The torque wrench included in the delivery scope of the Ceramill Motion 2 is appropriately set.

- ▷ Tighten fastening screw [1] with the torque wrench.

4.4 Starting the Grinding Program

- ▷ Briefly press operation button [A].
The machine advances to the starting position.
- ▷ Mount the transparent splash guard [1] to the extraction opening and lock it in place with lever [3].
- ▷ Transfer the milling file for wet processing.

5 Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

6 Download Infos

Further instructions/manuals and video tutorials can be downloaded under www.ceramill-m-center.com and/or www.amanngirrbach.com.

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Amann Girrbach AG
Amann Girrbach GmbH