



US DENTAL DEPOT

US DENTAL DEPOT

6555 NW 9th Avenue suite 212
Fort Lauderdale 33309
Florida USA

USA - 1 1866 625 5558 – 1 954 874 6325

Colombia 57 1 489 9051

Peru 51 1 707 3444

Venezuela 58 212 729 1639

Argentina 54 11 5258 7148 Cba 54 351 569 1409

www.USDENTALDEPOT.com



DUPLICATING DISPENSER MANUAL

START TO WORK WITH DUPLICATING DISPENSER

1. ADD 1GALLONG AND 1QUART OF ANTIFREEZE LIQUID INSIDE THE BASKET
2. CUT THE PASTE INTO LITTLE PIECES AND INTRODUCE INTO THE TANK TO OBTAIN BETTER RESULTS
3. CONNECT THE MACHINE TO THE MOTOR AT THE REAR PART OF THE PANEL
4. THE CONTROLS ARE HIGH TEMPERATURE AT 195 DEGREES FAHRENHEIT AND LOW TEMPERATURE OF 120 DEGREES FAHRENHEIT
5. RIGHT AFTER START TO ROTATE MELTING CYCLE CLOCKWISE