# BioSonic<sup>®</sup> S1

## Piezo Ultrasonic Scaling System

- Ⓔ Owner's Guide
- Ⓔ Guia de uso
- Ⓔ Manual do usuário
- Ⓔ 使用者指南

## BioSonic® S1 Piezo Ultrasonic Scaler

**CAUTION:** Federal law restricts this device to sale by, or on the order of, a licensed healthcare practitioner.

**An essential part of your periodontal and prophylaxis therapy, as well as, root canal treatment.**

Congratulations on your purchase of a Coltène/Whaledent BioSonic® S1 Piezo Ultrasonic Scaler. Developed specifically for use by dental hygienists, dentists, periodontists, and endodontists, Coltène/Whaledent's BioSonic® S1 Piezo Ultrasonic Scaler (here-in-after referred to in this manual as the "unit") is uniquely designed to save time by being a multi-functional device for general scaling as well as perio and endo dental procedures.

### What is scaling?

Scaling is the process by which calculus is removed from the tooth surface. This can be accomplished using either hand instruments (such as hand scalers or curettes) or ultrasonic scalers. When using ultrasonic scalers, high frequency waves generated within the ultrasonic handpiece cause the tip to vibrate at 28,000 cycles per second. This vibration removes calculus from tooth surfaces more rapidly and effectively than hand instruments, while also eliminating unhealthy biofilm.

### Piezo Ultrasonic Applications

- Dental Calculus Elimination
- Periodontal Therapy
- Root Canal Treatment

### I. Unpacking


- Remove the unit** from the shipping carton and inspect it for any damage which may have occurred during shipping.
- Make sure all parts and accessories** have been included according to the packaging list in Section VII.
- Check the serial number** on the underside of the unit against the number on the shipping carton. If these numbers are not the same, contact your dealer.
- Complete the warranty card** and mail to Coltène/Whaledent within 10 days of receipt.
- Position the unit** for easy access during patient treatment and also near a wall receptacle and cold water line. Make sure that the unit is placed on a flat surface, such as a countertop.

### II. Set Up

#### A. Water (H<sub>2</sub>O) Line Connection

- Incoming cold water supply line pressure to the unit must be a maximum of 50 PSI (0.01-0.50 MPa).
- The cold water line should be thoroughly flushed prior to connection of the unit (**See Section III. B. 1-4**).
- Turn the Water Control Knob to the maximum position at the back of the unit (**Fig. 1**).
- Connect the cold water supply line to the back of the unit and tighten with the connector (**Fig. 1**).
- Fasten the other end to the cold water supply line using the metal quick-connector.
- Inspect all connections to make certain there are no leaks.

Figure 1: Back of Unit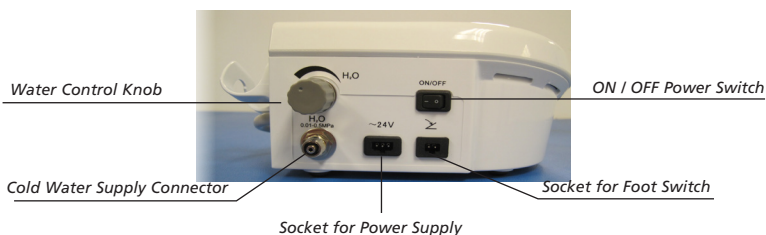

#### B. Power Cord/Foot Switch

- Plug the proper end of the Power Cord into the Power Supply Socket located at the back of the unit (**Fig. 1**).
- Insert the grounded plug supplied with the unit into a properly grounded electric receptacle, preferably "Hospital Grade."
- Insert the Foot Switch cable into its designated socket at the back of the unit (**Fig. 1**).
- Place the Foot Switch in a convenient place on the floor.
- Turn on the unit using the ON/OFF Power Switch located at the back of the unit. The General indicator (G) and the number 5 power level will light up.

#### C. Handpiece Connection and Components

- Attach the Handpiece to the Connector Cable (**Fig. 2**).
- Be sure to keep the Connector Cable and Handpiece interface dry when connecting.


#### Components:

**Nipple:** The Nipple can be removed by unscrewing it from the Handpiece. The shaft can be cleaned with isopropyl alcohol.

**Handpiece:** The Handpiece is the main component and can be autoclaved to 132°C.

**Connector Cable:** Connects the Handpiece to the water source and power supply of the unit.

Figure 2: Handpiece

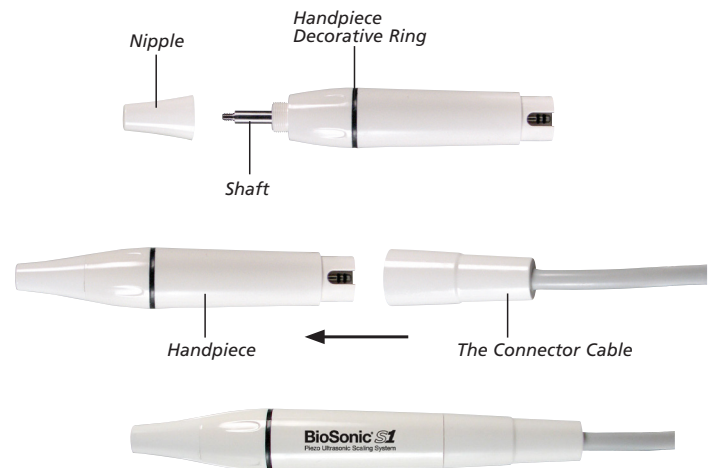
### III. Operating Instructions

#### A. Patient preparation and positioning

- Have the patient perform an antimicrobial pre-procedural mouth-rinse for up to 60 seconds to reduce bacterial contamination of the aerosol.
- Place the dental chair in a supine position for optimal access to both the maxillary and mandibular arches. This improves patient comfort and increases clinician visibility.
- Position the patient's head for optimum access to the quadrant and tooth surface being treated. Evacuate water and debris with a saliva ejector or high volume evacuator.

**Note:** Refer to section IV (Maintenance and Infection Control) of this Owner's Guide for general procedures to follow daily, between patients and at the end of the day.

Figure 3: BioSonic® S1

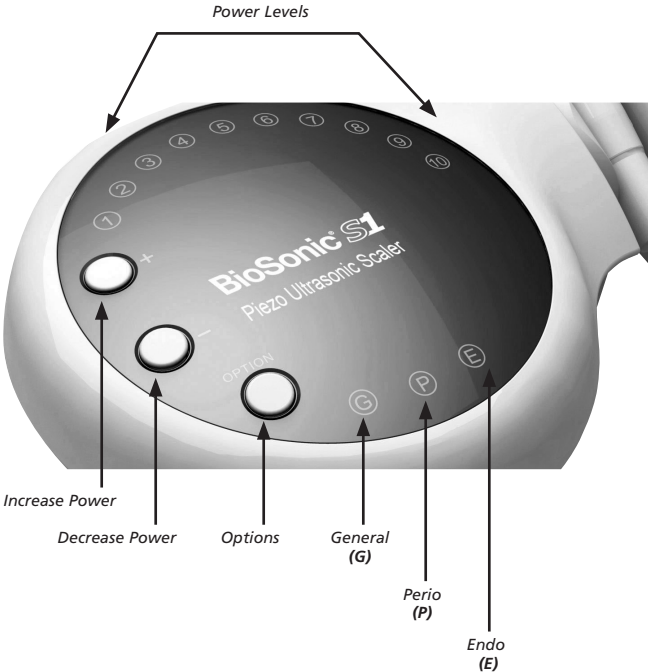B. General scaling procedures

1. Open the main shut-off valve on the dental office water supply.
2. Turn the unit ON using the ON/OFF Power Switch. The power level will be at the number 5 position (**Fig. 3**).
3. Make sure the Water Control Knob is turned to the maximum position (**Fig. 1**).
4. Hold the Handpiece (without the tip screwed on) over a sink or drain and flush thoroughly.
5. Select a suitable BioSonic® Piezo Scaler Tip which has been sterilized. Screw on the tip by aligning the threaded end of the tip to the threaded connection at the top of the handpiece. Tighten the tip with the supplied Tip Wrench until tight. According to the tip chosen, use the suggested power level (**See Section VIII B.**).
6. Step on the Foot Switch. The tip will begin to vibrate. Turn the Water Control Knob until a fine spray is formed in order to cool down the Handpiece and rinse the teeth.
7. The water and power levels can be adjusted during clinical treatment according to a patient's sensitivity and the rigidity of the gingival tartar.
8. There are 10 different power levels from which to choose with the BioSonic® S1 Piezo Ultrasonic Scaler.
9. Three pre-set options are also available:  
General (G) – Level 5  
Perio (P) – Level 3  
Endo (E) – Level 1
10. After treatment, keep the unit running in order to rinse off the tip.

**WARNING!**  
(Failure to comply may result in personnel injury and/or equipment damage.)

Ultrasonic instruments are subject to breakage at any time. To avoid patient injury do NOT use tips that are bent, altered or worn more than 2 mm. During the procedure avoid applying too much force or operating at an excessive power setting.

Do not disconnect the Handpiece while depressing the Foot Switch

C. PIEZO TIP WRENCH INSTRUCTIONS

1. Tip Installation: Hold the Handpiece and tighten the BioSonic® Piezo Scaler Tip with the Tip Wrench supplied, in a clock-wise direction.
2. Tip Removal: Hold the Handpiece and untighten the tip using the Tip Wrench in a counter-clock-wise direction.

- Note:** Sterilize the tip after every use.
- Note:** The Tip Wrench must be cooled to room temperature after sterilization.
- Note:** Keep the Tip Wrench in a cool, dry, ventilated area.

D. Piezo Endo Tip procedures

1. Screw the Endo Tip to the Handpiece using the Endo Wrench.
2. Unscrew the Screw Cap from the Endo Tip. Be careful not to loosen the small O-ring.
3. Place the Endo File into the Endo Tip hole.
4. Screw down the Screw Cap using the Endo Wrench to tighten the file.
5. Press the option button on the front of the unit until the Endo Indicator (E) is lit. The power level will be at number 1.

 <p><b>WARNING!</b> (FAILURE TO COMPLY MAY RESULT IN PERSONNEL INJURY AND/OR EQUIPMENT DAMAGE.)</p>	Endo Tip and Screw Cap must be fully tightened.
	Do not step on the Foot Switch until the file is inside the root canal
	The power range should not exceed level number 5 when using an Endo Tip.

E. Helpful hints

- Use the lowest effective power setting possible when scaling.
- If the Handpiece or tip becomes hot to the touch, there is either too little water, too much power or both. Adjust power and/or water settings accordingly.
- Periodically check the BioSonic® Piezo Tips for wear. A wear guide is included in this package.
- It is best to hold the Handpiece in a pen grasp using a very light touch when scaling both supra and subgingival.
- There are two rubber O-rings at the Handpiece connection. Lubricating the O-ring with a small amount of Petroleum Jelly could prolong the life of the O-rings.
- Periodically check and inspect the tips, Power Cords, water lines, connections, and Handpiece for wear.
- If sensitivity persists, alternate treatment of the sensitive areas and/or decrease the power setting.

IV. MAINTENANCE AND INFECTION CONTROL

A. Before each patient

1. Use a sterilized Handpiece.

2. Clean and disinfect the surfaces of the unit, Power Cord, Foot Switch Cord, Handpiece and Connector Cable **(See Section IV. B. 6).**
3. Flush the water supply lines thoroughly **(See Section III. B. 1-4).**
4. Clean, disinfect, sterilize and inspect tips before each use **(See Section IV. B. 4).**

#### B. At the end of each day

1. All BioSonic® Piezo Scaler Tips, Endo Tips, Tip Wrench, and Endo Wrench can be cleaned with Coltène/Whaledent's BioSonic® Ultrasonic Cleaners as a first step in the sterilization process.
2. Detach the Handpiece, remove the Piezo Tip from the Handpiece, clean and sterilize accordingly.
3. Make sure the water is removed from the Handpiece with compressed air before sterilization.
4. The Handpiece as well as the Piezo Tips, Endo Tips, Tip Wrench and Endo Wrench can be autoclaved at 132°C/2.2bar (0.22MPa).

**Note:** Ensure that the Handpiece is completely disassembled before autoclaving.

5. Allow the components to cool to room temperature before using again.
6. Clean and disinfect the unit and peripherals before use on each patient. Clean components by wiping with a soft cloth moistened with a commercially available mineral deposit remover followed by an intermediate level disinfectant.

	<p>Do not use petroleum based solvents, iodophors, or phenolic based products to clean components. (Iodophors and phenolics can stain the surface of the unit.) Clean up all liquid spills immediately.</p>
<p><b>CAUTION!</b></p>	

7. Flush the water lines thoroughly at the end of the day **(See Section III. B. 1-4).**
8. Close the main shut-off valve from the dental water supply and turn the power off.

**C. Water Filter Maintenance** – The stainless steel filter on the water line assembly should be cleaned periodically to prevent restricted water flow.

1. Unscrew water filter housing and wash stainless steel filter in warm water.
2. Re-assemble water filter housing.

#### V. Environmental Conditions

- Intended for indoor use only.
- Maximum altitude: 2000m
- Temperature range: -10°C - 50°C
- Maximum relative humidity: 80% for temperatures up to 31°C, decreasing linearly to 50% relative humidity at 40°C.
- Degree of protection against harmful ingress of water:
  - Ordinary equipment: IPX0
  - Foot Switch: IPX1
- Degree of safety in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide: Not suitable.

#### VI. Warnings

<div style="text-align: center;">  </div> <p><b>WARNING!</b> (Failure to comply may result in personnel injury and/or equipment damage.)</p>	Use this product in accordance with the instructions contained in this user's manual.
	Ultrasonic instruments are subject to breakage at any time. To avoid patient injury do NOT use tips that are bent, altered or worn more than 2 mm. During the procedure avoid applying too much force or operating at an excessive power setting.
	For professional use only.
	Use cold water only.
	Do not use the BioSonic® S1 Piezo Ultrasonic Scaler without water.
	Avoid touching the metal working tip of an activated BioSonic® S1 Piezo Ultrasonic Scaler with bare hands, lips, gums, or tongue as it may cause minor burns.
	Do not use the ultrasonic scaler for dental procedures involving the condensation of amalgam.
	To assure optimum performance, use only BioSonic® Piezo Tips.
	This equipment generates, uses and can radiate radio frequency energy. If not installed and used in accordance with the instruction manual, it may cause interference to radio communications. Operation of this equipment in a manner inconsistent with the manual is likely to cause interference in which case the user, at his/her own expense, will be required to take whatever measures may be required to correct the interference.
	Interference may occur in the vicinity of equipment marked with the following symbol:
	
	Persons fitted with cardiac pacemakers have been cautioned that some types of electronic equipment might interfere with the operation of a pacemaker. Although no instance of pacemaker interference relating to a BioSonic® S1 Piezo Ultrasonic Scaler has ever been reported to Coltène/Whaledent Inc., we recommend that the Handpiece and cables be kept at least 15.24 – 22.86 cm away from any pacemaker or pacemaker leads during use. If use of this product is in question, consult the patient's cardiologist.
	This device should not be used on patients with hemophilia, without the advice of the patient's physician.
	Extreme caution is required when using this device around patients with heart disease, pregnant women, or small children.
	Do not restrict air flow. Provide adequate ventilation.
	Do not subject unit to shock or impact.
	Do not immerse unit in water.

 <p><b>WARNING!</b> (Failure to comply may result in personnel injury and/or equipment damage.)</p>	Unspecified or improper use of this product may impair safety protection.
	Do not place the unit on or next to a radiator or other heat source. Excessive heat may damage the unit's components.
	If the device is not used for an extended period of time, turn it on once per month for five minutes.
	Dispose of tips that have been bent, altered, or worn in any way.
	Use retraction to protect the tongue, cheek and lips when the scaler is in use to avoid contact between the two.
	The normal frequency of the unit is extremely high, therefore, when operating the tip, touch the tooth lightly in a side-to-side motion. This will eliminate any calculus build-up. During clinical treatment, do not apply the tip vertically. This may harm a patient's tooth, or cause damage to the tip. Overexertion and constant scaling in a specific spot can cause heat and pain for the patient. The use of a higher than recommended power level can cause damage to the tip as well.

## VII. Order Information

### Units:

Catalog No.	Line Cord
USP100A	USA (Type A)
USP100C	EURO (Type C)
USP100G	UK (Type G)
USP100IA	AUSTRALIA (Type I)

### Packaging List:

Includes: Main Unit, Detachable Handpiece, Universal Power Supply, Outlet Cord, Foot Switch, 6 Tips with Wear Guide, Tip Wrench, Endo Wrench, Water Hose/Filter, 4 – Waterproof O-Rings, Tip Holder

### Handpiece Tips – S - Select Series

Catalog No.	Category
USS1S	GENERAL SCALING
USS2S	GENERAL SCALING
USS3S	GENERAL SCALING
USS4S	GENERAL SCALING
USS5S	GENERAL SCALING
USS6S	GENERAL SCALING
USP1S	PERIO
USP2S	PERIO
USP3S	PERIO
USP4S	PERIO
USP5S	PERIO
USP6S	PERIO
USP7S	PERIO
USP8S	PERIO
USE1S	ENDO
USE2S	ENDO
USE3S	ENDO
USE4S	ENDO
USE5S	ENDO
USC1S	CAVITY PREP
USC2S	CAVITY PREP
USC3S	CAVITY PREP
USC4S	CAVITY PREP
USC5S	CAVITY PREP

## VIII. Specifications

### A. Unit

Power Input	100V - 240V - 50/60 Hz 1.2A (max)
Main Unit Input	24 VAC 1.3A
Output Power	0.06W - 20W
Frequency	28kHz ± 3 kHz
Water Supply Pressure	0.01 - 0.50 MPa
Net Weight	4.25 lbs. (1.9 kg)

### B. Handpiece Tips

Catalog No.	Suggested Power Level
USS1S	Low/Medium
USS2S	Low/Medium
USS3S	Low/Medium
USS4S	Low/Medium
USS5S	Low/Medium
USS6S	Low/Medium
USP1S	Low
USP2S	Low/Medium
USP3S	Low/Medium
USP4S	Low
USP5S	Low
USP6S	Low
USP7S	Low
USP8S	Low
USE1S	Low
USE2S	Low
USE3S	Low/Medium
USE4S	Low
USE5S	Low
USC1S	Low/Medium
USC2S	Low/Medium
USC3S	Low/Medium
USC4S	Low/Medium
USC5S	Low/Medium

## IX. Symbols

Class II Equipment

Caution, consult accompanying documents

Type BF equipment

Indoor use only


Alternating current

Can be autoclaved

Socket for the Foot Switch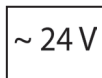


24VAC power supply socket

Adjustment for the water flow

Water entrance pressure  
0.01 MPa - 0.5 MPa

Power Switch

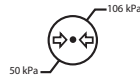Atmospheric pressure for storage

Temperature limitation

Appliance compliance WEEE directive

Humidity limitation

Authorized Representative in the European Community

Non-ionizing electromagnetic radiation

## X. Troubleshooting

Problem	Possible Causes	Solutions
The scaling tip does not vibrate and no water flows out when Foot Switch is pressed	The power line plug is loose	Insert plug into socket
	The Foot Switch connection is loose	Insert Foot Switch connection into its socket
The scaling tip does not vibrate, but water flows out when foot switch is pressed	The tip is loose	Screw the tip onto the Handpiece tightly
	Handpiece Issues	Contact us
The scaling tip vibrates, but there is no spray when the Foot Switch is pressed	The Water Control Knob is at a minimal setting	Increase the water supply using the Water Control Knob
	The water system is blocked	Clean the water line and filter
There is still water flowing after the power is turned off	There is a blockage in the electro-magnetic valve	Contact us
The Handpiece generates heat	The water control knob is at a minimal setting	Increase the water supply using the control knob
The amount of spouting water is minimal	The water pressure is not high enough	Increase the water pressure
	The water line is blocked	Clean the water line and the filter
The vibration of the tip is weak	The tip is loose	Screw the tip onto the Handpiece tightly
	The tip loosened due to the vibration	Screw the tip onto the Handpiece tightly
	The connection between the Handpiece and the cable is not dry	Dry the connection using hot air
	The tip is damaged or worn	Replace the tip with a new one
There is water leaking from the Handpiece/Cable Connection	One of the O-rings is damaged	Replace the damaged O-ring
The file does not vibrate	The Screw cap is loose	Screw on the Screw Cap tightly
	The Endo Tip is damaged	Replace the damaged Endo Tip
There is noise coming from the Endo Tip	The Screw Cap is loose	Screw on the Screw Cap tightly

## XI. Limited Warranty

Our products are carefully manufactured to meet stringent quality assurance requirements. Our products are manufactured from new parts or new and serviceable used parts. Regardless, our warranty terms apply. This product has been developed specifically for use in dentistry and is intended to be operated only by qualified dental professionals in accordance with the instructions contained in this guide. However, notwithstanding anything contained herein to the contrary, the user shall at all times be solely responsible for determining the suitability of the product for the intended purpose and the method of its use. Any guidance on application technology offered by or on behalf of the manufacturer, whether written, verbal or by demonstration, shall not relieve the dental professional of his/

her obligation to control the product and to make all professional judgments regarding its use.

Our products are warranted in accordance with the terms of a written Certificate of Limited Warranty accompanying each product. Except for the warranties specifically set forth in the Certificate of Limited Warranty, Coltène/Whaledent Inc. provides no warranties or guarantees of any kind covering the product, expressed or implied, including, without limitation, any warranties as to merchantability or fitness for a particular purpose. **The purchaser/user is referred to the Certificate of Limited Warranty for all of the terms, conditions and limitations of the warranty covering this product.** This Section of the user manual is not intended to in any way modify or add to the warranty provided in the Certificate of Limited Warranty.

Any claim for damage or breakage to the product in transit should be made to the carrier promptly upon discovery. C/W does not warrant the product against shipping damage.

**Note:** BioSonic® Piezo Tips have a 90 day warranty period.

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**Coltène/Whaledent AG**
**Coltène/Whaledent GmbH + Co. KG**