## PRODUCT DESCRIPTION

BeautiCem Veneer is a radio-opaque, color stable, light-cured resin cement intended to cement laminate veneers, inlays and onlays to natural dentition. Use BeautiBond Universal Adhesive (a self-etching adhesive) to modify tooth, alumina, zirconia surface and BeautiBond Universal CR Enhancer to prime the adhesive surface of the restoration fabricated with ceramics (silicate glass, lithium silicate glass) or composite resin. BeautiCem Try-in Paste is a trial paste to confirm suitable paste shade before placing the restoration. It is water-soluble and can be easily removed after confirming the shade.

## INDICATIONS FOR USE

Cementation of ceramic or composite resin laminate veneers and translucent inlays and onlays with a thickness of less than 2 mm

## PRECAUTIONS AND WARNINGS

- Do not use this product on patients with known allergies to this material and/or methacrylate monomer.
- Operators with known allergies to methacrylate monomer must not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- Use protective medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers can penetrate through the gloves very quickly to reach the skin. Therefore, should this material come in contact with the gloves, immediately remove them and wash hands with plenty of water.
- In case of contact with eyes, flush immediately with plenty of water and seek medical advice.
- Follow the instructions for use of each dental material, instrument or equipment to be used in conjunction with this product (eg. BeautiBond Universal and BeautiCem Try-in Paste).
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package. (Example ☐ YYYY-MM-DD → Year-Month-Day of the expiration date)
- This product is intended for use by dental professionals only.

## DIRECTIONS FOR USE

### Preparation

When storing these materials in a refrigerator, allow them to reach room temperature before use. Attach the needle tip to the syringe as indicated in the figures below.

- Pull off the shield cap from the syringe while turning it slowly. (Fig. A)
- Attach the needle tip by turning it clockwise until it is securely attached. (Fig. B)
- Insert the needle tip into the small hole on the head of the shield cap immediately before use to protect the material from ambient light. (Fig. C)
  - When replacing the needle tip, always use clean medical (dental) gloves.

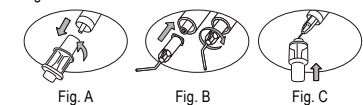
Fig. A Fig. B Fig. C

### Application

1. Preparation of indirect restoration  
Roughen the restoration surface to be cemented with an abrasive or sandblast with 50-100 µm alumina sand at a prescribed pressure described in Table 1. Then, rinse with water and dry.

Adhesive surface	Pressure
Ceramics (silicate glass, lithium silicate glass) Composite resin	Approx. 0.1 - 0.2 MPa (Approx. 1 - 2 Kg/cm <sup>2</sup> )
Ceramics (alumina, zirconia)	Approx. 0.2 - 0.3 MPa (Approx. 2 - 3 Kg/cm <sup>2</sup> )

- Refer to the manufacturer's instructions for use to select optimal pressure.
- Removal of temporary sealing or provisional restoration  
Thoroughly remove the temporary sealing or provisional restoration. Rinse with water and dry following the standard method.
  - Do not use eugenol containing materials since they may inhibit polymerization or adhesion of the material.

3. Try-in the indirect restoration (BeautiCem Try-in Paste)  
Select the proper shade of BeautiCem Try-in Paste and apply to the adhesive surface of the restoration. Place the restoration onto the tooth to confirm the shade. After try-in, thoroughly rinse off the Try-in Paste remaining on the tooth and restoration surface. In case of shade mismatch, try with another shade of Try-in Paste until you obtain the optimal shade.

4. Priming of the indirect restoration  
Dry the adhesive surface completely while confirming that there is no residual BeautiCem Try-in Paste on the surface. Prime the restoration surface with the suitable primer based on the substrate to be bonded (eg. BeautiBond Universal CR Enhancer, SHOFU Porcelain Primer etc.).

- Refer to the manufacturer's instructions for use of the material of the indirect restoration for appropriate priming.
- Pretreatment of the tooth
  - Cleaning of the tooth surface  
Dry the tooth thoroughly and confirm that there is no residual Try-in Paste on the tooth surface. Rinse with water and dry.
  - Isolation  
Use a rubber dam etc to isolate the restoration site.
  - Etching  
Etch tooth surface with etching material, when necessary (eg. SHOFU Enamel Conditioner, FL-BOND II Etchant). Then, rinse thoroughly with water and dry to remove the etching material.
  - Application of BeautiBond Universal Adhesive  
Dispense adequate amount of BeautiBond Universal Adhesive onto a disposable dish and apply to the tooth surface with a disposable brush. Leave undisturbed for 10 sec. Air dry with gentle air for 3 sec. and then, dry with stronger air to dry the surface.
    - Use all the dispensed material only on one patient to avoid cross infection.

Do not reuse disposable brush and disposable dish with cross contamination.

- Light-curing  
Light-cure with a light-curing unit; Halogen: 10sec./LED: 5 sec. In the case where the adhesive surface is large, light-cure in segments.
- Placement/Seating of the indirect restoration
  - Application of BeautiCem Veneer  
Apply BeautiCem Veneer onto the adhesive treated restoration surface and place it on the natural tooth and apply gentle pressure to allow excess cement to extrude from the margins.
    - Insertion of polyester matrix between the prepared abutment and adjacent teeth prior to placement of the restoration aids in isolation and easy removal of excess cement.
    - Needle tip should be used for only one patient and then discarded at the end of the procedure to avoid cross contamination.
    - Disinfect the syringe and shield cap with alcohol after each use to avoid cross-contamination. Then, tightly close the cap. Use caution to avoid alcohol contamination of the paste.
  - Removal of excess paste  
Tack-cure the paste for 2-3 sec. and remove the excess with an appropriate instrument while applying gentle pressure onto the restoration. Avoid applying excessive pressure to the restoration when removing the excess material.
  - Light-curing  
Light-cure the cement from labial, lingual and interproximal surfaces with a light-curing unit; Halogen for 20 sec./LED for 10 sec. for each surface. In the case where restored area is large, light-cure in segments.
- Finishing and Polishing  
Finish the margin with rotary instruments such as ZiLMaster Fine, CeraMaster (for ceramic), CompoMaster (for composite resin) etc. following the standard method. Adjust the occlusion if necessary.

Light-curing unit	Halogen	LED
Light curing time		
BeautiBond Universal Adhesive	10 sec.	5 sec.
BeautiCem Veneer	20 sec.	10 sec.
Wavelength	400 - 500 nm	440 - 490 nm
Light intensity	≥ 500 mW/cm <sup>2</sup>	≥ 1000 mW/cm <sup>2</sup>

## COMPOSITION

Glass powder, Bis-GMA, Polymerization initiator, pigments and others

## SHADES

L-Value (Low Value), M-Value (Medium Value), H-Value (High Value), Ivory-L (Ivory-Light), Ivory-D (Ivory-Dark)

## STORAGE

Store at room temperature (1-30 °C/34-86 °F) in dark place. Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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