# VINTAGE PRO POWDER OPAQUE LIQUID

- EN

Instructions for use

DE

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FR

Mode d'emploi

NL

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ES

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IT

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EN

READ CAREFULLY BEFORE USE

Mixing Liquid for VINTAGE PRO Powder Opaque

## PRODUCT DESCRIPTIONS

VINTAGE PRO POWDER OPAQUE LIQUID is dedicated for VINTAGE PRO Powder Opaque. VINTAGE PRO POWDER OPAQUE LIQUID-L (L-type liquid), provides longer working time than standard liquid (VINTAGE PRO POWDER OPAQUE LIQUID) by keeping the mixture optimally consistent.

## INDICATIONS FOR USE

Mixing liquid for VINTAGE PRO Powder Opaque for obtaining creamy consistency

## PRECAUTIONS AND WARNINGS

- Do not handle the liquid or the mixture of this material with bare hands. This product should not come in contact with eyes or skin. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- If any inflammation or other allergic reactions occur on the operator while using this product, immediately discontinue use and seek medical advice.
- Avoid any source of ignition since this product is flammable.
- Use this product in well ventilated areas (several appropriate ventilations per hour are recommended).
- Do not leave these materials in areas of high temperature, such as near the heater etc. Avoid direct sunlight.
- Tightly close the cap after each dispensing.
- Do not use VINTAGE PRO POWDER OPAQUE LIQUID in conjunction with any materials other than VINTAGE PRO Powder Opaque.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- This product is intended for use by dental professionals only.

## DIRECTIONS FOR USE

- Wash Opaque**  
Mix VINTAGE PRO Powder Opaque with powder opaque liquids until it reaches thin consistency. Apply the mixture to the clean, dry substructure with a brush.
- Opaque**  
Mix VINTAGE PRO Powder Opaque with powder opaque liquids until it reaches creamy consistency. Apply the mixture with a brush or instrument to completely mask the surface to be veneered.  
➤ Use L-type liquid to extend working time.

## COMPOSITION

VINTAGE PRO POWDER OPAQUE LIQUID 1,3-Butanediol  
VINTAGE PRO POWDER OPAQUE LIQUID-L Glycerin, Propylene glycol, 1,3-Butanediol

## STORAGE

Tightly close the cap and store at room temperature.  
Keep away from direct sunlight and any source of ignition.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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