# VINTAGE PRO PASTE OPAQUE LIQUID

- EN

Instructions for use
- DE

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Instrucciones de uso
- IT

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**EC REP**  
SHOFU DENTAL GmbH  
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

**EN** READ CAREFULLY BEFORE USE

## Mixing Liquid for VINTAGE PRO Paste Opaque

**PRODUCT DESCRIPTIONS**  
VINTAGE PRO PASTE OPAQUE LIQUID is dedicated for VINTAGE PRO Paste Opaque.

**INDICATIONS FOR USE**  
Mixing liquid for dilution of VINTAGE PRO Paste Opaque to obtain optimal creamy consistency

- PRECAUTIONS AND WARNINGS**
- Do not handle the liquid or the mixture of this material with bare hands. This product should not come in contact with eyes or skin. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
  - If any inflammation or other allergic reactions occur on the operator while using this product, immediately discontinue use and seek medical advice.
  - Avoid any source of ignition since this product is flammable.
  - Use this product in well ventilated area (several appropriate ventilations per hour are recommended).
  - Do not leave this material in areas of high temperature, such as near the heater etc. Avoid direct sunlight.
  - Tightly close the cap after each dispensing.
  - Do not use VINTAGE PRO PASTE OPAQUE LIQUID in conjunction with any materials other than VINTAGE PRO Paste Opaque.
  - Do not use this product for any purposes other than specifically outlined in these instructions for use.
  - This product is intended for use by dental professionals only.

- DIRECTIONS FOR USE**
- Wash Opaque**  
Mix VINTAGE PRO Paste Opaque with VINTAGE PRO PASTE OPAQUE LIQUID until it reaches thin consistency. Apply the mixture to the clean, dry substructure with a brush.
  - Opaque**  
If required, mix VINTAGE PRO Paste Opaque with VINTAGE PRO PASTE OPAQUE LIQUID until it reaches creamy consistency. Apply the mixture with a brush or instrument to completely mask the surface to be veneered.

**COMPOSITION**  
Glycerin, Propylene glycol, 1,3-Butanediol

**STORAGE**  
Tightly close the cap and store at room temperature.  
Keep away from direct sunlight and any source of ignition.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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