



Venus White

Make every day a Venus White day.

Venus White's three levels of whitening offer customized treatment, enhanced flexibility and superior results. Increase production and patient satisfaction with products designed to help patients achieve bright, radiant smiles.



Giving a hand to oral health



KULZER
MITSUI CHEMICALS GROUP

Personalized professional whitening

Three levels of whitening



1 Venus White Max

In-Office Whitening

- Three or four 15-minute treatments with 38% hydrogen peroxide gel
- Syringe-to-syringe mixing chemically activates the product, ensures a thorough mix and provides maximum freshness
- Contains potassium nitrate, which helps decrease sensitivity associated with bleaching
- To keep teeth white, follow with Venus White Pro custom take-home trays or Venus White Ultra pre-filled, disposable whitening trays



2 Venus White Pro

Take-Home Whitening for Custom Trays

- Custom whitening trays take a patient's smile up to 8.4 shades whiter with virtually no sensitivity*
- Contains potassium nitrate, which helps decrease sensitivity associated with bleaching
- 3 strength levels available: 16%, 22%, and 35% Carbamide Peroxide.
- Sticky, viscous gel is mint flavored for patient comfort
- Full Patient Kits, Refill Kits and 50-Syringe Packs available
- To keep teeth their whitest, Refill Kits should be used for touch-up every 6–12 months

*Results may vary.

Venus White Pro is proven to whiten teeth up to 8.4 shades with virtually no sensitivity!*


The University of Texas

Venus White Pro Effectiveness Study Results
Shade Change Measurement, Canine

	At 2 Weeks	At 3 Weeks	At 4 Weeks
Shade Guide Units (Vita Classical)	8.4	8.0	8.0

Venus White Pro Sensitivity Study Results

Average sensitivity ratings below "2" representing virtually no sensitivity

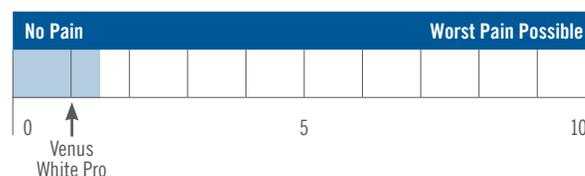



3 Venus White Ultra

Pre-Filled, Disposable Whitening Trays

- Convenient entry-level whitening provides unique business opportunities and opens doors to more advanced options
- Each disposable tray is pre-filled with mint-flavored 11.2% or 15% hydrogen peroxide gel
- Once-a-day treatment for up to seven days
- Visibly whiter teeth in just three days
- No chair time or multiple visits required
- Ultra thin—designed to mold securely around each tooth for maximum comfort

See the Venus White Difference!



Before

Venus White Pro Patient.
Dentistry by Shannon Pace Brinker, CDA CDD.



After

Venus White personalized professional whitening

Venus White Max In-office Whitening		Venus White Pro Take-home Whitening for Custom Trays	
40005211	In-Office Whitening Kit (2 x 0.96ml Hydrogen Peroxide, 2 x 0.24ml Activator, 1 x 1.2ml Gingival Barrier, 5 x 22ga Flock Tips, and 2 x 20ga Black Mini Tips)	40005166	16% Patient Kit (6 x 1.2ml Syringes, Tray Case, Shade Guide)
40005202	Light Cure Gingival Barrier (4 x 1.2ml Syringes, 8 Black Mini Tips)	40005167	22% Patient Kit (6 x 1.2ml Syringes, Tray Case, Shade Guide)
40005240	Venus Comfort Gel (4 x 1.2ml Syringes, 4 Tips)	40005460	35% Patient Kit (6 x 1.2ml Syringes, Tray Case, Shade Guide)
Venus White Ultra Pre-filled, Disposable Whitening Trays		40005164	16% Refill Kit (3 x 1.2ml Syringes)
		40005165	22% Refill Kit (3 x 1.2ml Syringes)
		40005462	35% Refill Kit (3 x 1.2ml Syringes)
		40005463	16% 50-Syringe Pack (50 x 1.2ml Syringes)
		40005464	22% 50-Syringe Pack (50 x 1.2ml Syringes)
		40005465	35% 50-Syringe Pack (50 x 1.2ml Syringes)
		40005203	EVA Sheets (25 Sheets x 0.040" thick)

Customizable Patient Marketing Materials

Get the most out of Venus White with customizable marketing materials promoting your practice.
www.KulzerMarketingSolutions.com



Distributed by:
Kulzer, LLC
 4315 S. Lafayette Blvd.
 South Bend, IN 46614
 (800) 431-1785